

EDAN INSTRUMENTS, INC.



**Smart ECG-Viewer**  
**Data Management Software**

# User Manual

Manual Ver: V1.4

Release Date: Dec. 2007

Part Number: MS1R-20161-V1.4

# Copyright

Copyright ©EDAN INSTRUMENTS, INC. 2007. All rights reserved.

## Attention

EDAN INSTRUMENTS, INC. (hereinafter called EDAN) makes no warranty of any kind with regard to this material, including, but not limited to the implied warranties of merchantability and fitness for a particular purpose. EDAN assumes no responsibility for any errors that may appear in this document, or for incidental or consequential damage in connection with the furnishing, performance or use of this material.

No part of this document may be photocopied, reproduced or translated to another language without prior written consent of EDAN.

The information contained in this document is subject to change without notice.

## Responsibility of the Manufacturer

EDAN only considers itself responsible for any effects on safety, reliability and performance of the equipment if:

Assembly operations, extensions, re-adjustments, modifications or repairs are carried out by persons authorized by EDAN, and the electrical installation of the relevant room complies with national standards, and the instrument is used in accordance with the instructions for use.

## Warranty

EDAN warrants that the product sold to you is free from defects in materials and workmanship, and shall conform to its product specification as defined in the user manual.

In the status of normal operation and maintenance, if EDAN receives notice of such defects during the warranty period, EDAN shall, at its option, either repair or replace it without charge.

## Limit of Warranty

The charges of freightage and others aren't covered under warranty. Misuse, accident, modification, unsuitable physical or operating environment, improper maintenance, or damage caused by a product for which we are not responsible may void the warranty.

## Using This Label Guide

This guide is designed to give key concepts on safety precautions.

 **WARNING** 

A **WARNING** label advises against certain actions or situations that could result in personal injury or death.

 **CAUTION** 

A **CAUTION** label advises against actions or situations that could damage equipment, produce inaccurate data, or invalidate a procedure.

**Note:** A **NOTE** provides useful information regarding a function or procedure.

# Table of Contents

<b>1 Safety Guidance</b> .....	<b>1</b>
<b>2 Introduction</b> .....	<b>4</b>
2.1 Feature.....	4
2.2 System Running Environment .....	5
2.2.1 Hardware .....	5
2.2.2 Software .....	5
<b>3 System Installation and Connection</b> .....	<b>6</b>
3.1 Connection of the Main Unit.....	6
3.2 Install the Software .....	7
3.2.1 Install MSDE2000.....	7
3.2.2 Install the Software Protection Driver.....	8
3.2.3 Install Smart ECG-Viewer .....	10
3.2.4 Install PDF creator .....	14
<b>4 Operation</b> .....	<b>16</b>
4.1 Startup .....	16
4.2 Log In .....	16
4.3 Import Data from Harddisk.....	16
4.4 Receive ECG Data .....	17
4.5 View and Edit Patient and Record Information .....	21
4.5.1 New Patient Record.....	21
4.5.2 View and Modify Patient Record.....	23
4.5.3 Delete Patient Record.....	24
4.5.4 Assign Patient Record .....	24
4.5.5 Delete Case Record .....	25
4.5.6 View Case Record .....	25
4.5.7 Search Patient Record .....	25
4.6 Case Records Window .....	26
4.7 View ECG Curve.....	27
4.8 Rhythm Data Browse .....	30
4.9 Multi-Cases Rhythm Data Comparing Browse.....	32
4.10 Auto Measurement Parameter View.....	34
4.11 Auto Diagnoses Result View .....	35
4.12 Preview and Print Diagnosing Report.....	35
4.12.1 Display Format Setting .....	38
4.12.2 Page Speed Setting .....	40
4.12.3 Gain Setting.....	41
4.12.4 Rhythm Lead Select .....	42
4.12.5 Sequence Select.....	43
4.12.6 Print Content Select.....	44
4.12.7 Preview.....	45

4.12.8 Print .....	47
4.12.9 Create PDF .....	47
4.12.10 Print Table .....	48
4.13 User Management .....	51
4.13.1 New User.....	51
4.13.2 Delete User.....	52
4.13.3 Modify Password .....	52
4.14 Time Format Setting.....	52
4.15 User Defined Setting.....	53
4.16 Grid Color Setting.....	56
<b>5 Maintenance and Service.....</b>	<b>58</b>
5.1 Maintenance .....	58
5.2 Service Information.....	59

# 1 Safety Guidance

**The working conditions of the system are as follows:**

- Z** Environment Temperature: 10°C ~ 30°C
- Z** Relative Humidity: ≤70%
- Z** Atmospheric Pressure: 860hPa ~ 1060hPa

**The requirement for power:**

- Z** Power Supply: AC 220V
- Z** Power Frequency: 50Hz

**Symbol guide:**

 **Note! Refer to document**

To use the system safely and effectively, firstly be familiar with the operation method of Windows and read the user manual in detail to be familiar with proper operation method for the purpose of avoiding the possibility of system failure. The following **WARNING** and **CAUTION** precautions during the operation of the system must be paid more attention to.

 **WARNING** :

The system should be installed by a qualified service engineer. Do not switch on power until all cables have been properly connected and verified.

 **WARNING** :

The user of this system should get professional training and read this manual in detail before using it.

 **WARNING** :

This system is not explosion-proof and can not be used in the presence of flammable anaesthetic.

 **WARNING** :

Do not remove the unit covers for the purpose of avoiding getting an electric shock.

**⚠️WARNING⚠️:**

Do not move the main unit and monitor while power on.

**⚠️WARNING⚠️:**

Only EDAN supplied or recommended accessories can be connected to the system.

**⚠️WARNING⚠️:**

Ensure that the environment in which the system is operated is not subject to any sources of strong electromagnetic interference, such as radio transmitters, mobile telephones, etc. Keep them far away.

**⚠️CAUTION⚠️:**

The system should be used within temperature range from 10°C to 30 °C.

**⚠️CAUTION⚠️:**

Keep the environment clean. Avoid vibration. Keep it far from corrosive medicine, dust area, high-temperature and humid environment.

**⚠️CAUTION⚠️:**

The user must periodically check that whether the power cable is worn out or damaged. If damage is evident, replacement is recommended before use.

**⚠️CAUTION⚠️:**

Turn off the system power before connecting or disconnecting any accessory.

**⚠️CAUTION⚠️:**

Please do not operate the system before servicing or running normally.

**⚠️CAUTION⚠️:**

Turn off the system power and remove the power cable before servicing or maintaining the system.

**⚠️CAUTION⚠️:**

To this system, precautionary maintenance, including periodic cleaning and checking appearance can be finished by user because this maintenance does not touch the interior.

 **CAUTION** :

Avoid using attrite material to clean. Removing all dust from the exterior surface of the equipment with a soft brush or cloth or with a soft cloth, slightly dampened with a mild detergent solution or cool disinfectant. Especially the tie-in and panel edge should be noticed.

 **CAUTION** :

Avoid pouring liquids on the equipment while cleaning, and do not immerse any parts of the equipment into any liquids.



## 2 Introduction

Smart ECG-Viewer Data Management Software can store, display, analyze and manage ECG data on PC machine. It is the accessorial software of electrocardiograph.

Smart ECG-Viewer Data Management Software realizes great functions, including receiving and importing ECG data, managing patient information, displaying and measuring and comparing ECG average template and rhythm data, displaying and editing auto measurement parameter and auto diagnosis result, printing ECG wave and report, creating PDF file and so on. Consequently, the time spent on reading curve can be shortened, and the doctor's workload can be reduced greatly. Besides, the system manages the ECG data with database, and supports the relative history data searching and reviewing when necessary, which can assist the doctor to give decision.

### 2.1 Feature

- Z** Support transmitting ECG data through serial port or net port;
- Z** Support importing single data or data folder from hard disk of PC machine or U disk;
- Z** Great capability of database for data storage and management; Permanent storage for ECG data, patient information, auto measurement parameter and diagnosis result etc;
- Z** Support browsing 10s ECG wave; support viewing and measuring ECG mean and rhythm data with electronic ruler; support displaying auto measurement result and diagnosis result;
- Z** Support editing and modifying patient information, doctor information and diagnosis result;
- Z** Support preview and print, rich display and print format;
- Z** Support searching patient information through Patient ID, Name, Patient Number and Time etc;
- Z** Support that one patient has many case records; support multi-cases comparing browse;
- Z** Support creating ECG wave and other information into PDF format file;
- Z** Support time format setting, multi-user management etc;
- Z** Support 60s wave review in Rhythm1 mode or 20s wave review in Rhythm3 mode.

## 2.2 System Running Environment

### 2.2.1 Hardware

<b>CPU:</b>	PentiumIV2.4GHz or above
<b>Hard Disk:</b>	The size of hard disk should be chosen according to the intending amount of the records to be saved. The size of every record is approximately about 180KB.
<b>System Memory (RAM):</b>	256MB or above
<b>Communication Protocol:</b>	RS232, Ethernet port (one of them must exist at least)
<b>Display:</b>	17' TFT 1024×768 resolution, 32 bit actual color, regular icon and font setup
<b>Printer:</b>	Color ink jet printer or black-and-white laser printer
<b>Others:</b>	CD-ROM or DVD-ROM

### 2.2.2 Software

- Z** Windows2000 or Windows XP operating system
- Z** MSDE2000(Microsoft SQL Server 2000 Desktop Engine)

## 3 System Installation and Connection

### 3.1 Connection of the Main Unit

**⚠WARNING⚠:**

The system should be installed by a qualified service engineer. Do not switch on power until all cables have been properly connected and verified.

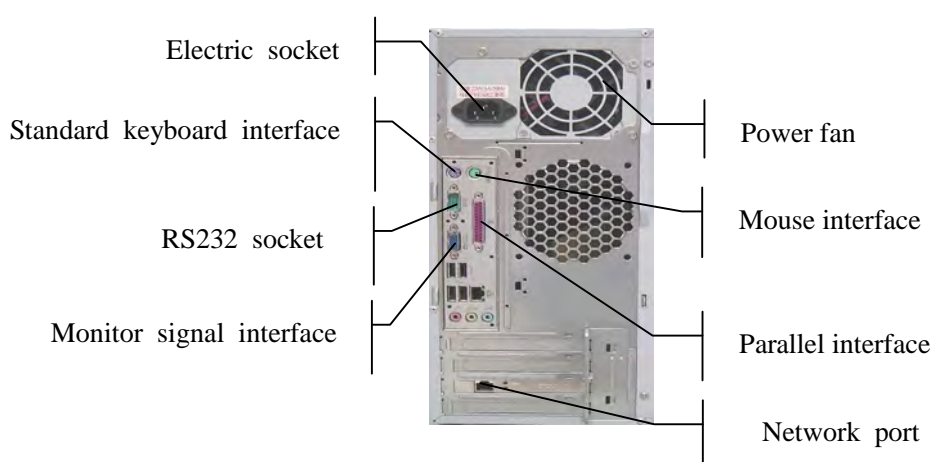


Figure 3-1 Rear panel of the main unit

**Note:** Figure 3-1 is just for reference. The position of corresponding interface may be different according to the actual main unit of the system.

#### **Connection of the main unit of central station (Refer to Figure3-1)**

- 1) Connect monitor with the main unit by signal cable;
- 2) Connect standard keyboard、 mouse with corresponding interfaces at the rear panel of main unit;
- 3) Insert the Softdog into parallel interface at the rear panel (For USB Softdog, it should be inserted into the USB interface);
- 4) Connect the RS232 socket at the rear panel of main unit with the RS232 socket of the electrocardiograph by RS232 cable;
- 5) Connect the network port at the rear panel of main unit with the network port of the electrocardiograph by network cable;
- 6) After make sure that the above connections are right, connect the main unit、 monitor and printer with 220V power supply.

## 3.2 Install the Software

The user should install the following software in sequence:

1. MSDE2000
2. Software Protection Driver
3. Smart ECG-viewer
4. PDF Creator (It is used to create PDF file. If the user does not need this function, this software and Adobe Reader software can not be installed.)
5. Adobe Reader (It is used to read PDF file. If the user needs this function, install it by yourself; if do not need this function, it can not be installed.)

**Note:** If it is the situation of upgrading the Smart ECG-Viewer software of old version, the MSDE2000 can not be installed again. When install the Smart ECG-Viewer software of new version, the old one should be uninstalled firstly, and the Installation Destination Folder of new version should be in accordance with that of the old one.

**Note:** For old version and new version, both the default Destination Folders are C:\Program Files\SmartECGViewer.

**Note:** To View PDF file, both the PDF Creator and Adobe Reader should be installed.


### 3.2.1 Install MSDE2000


#### 1. Installation

Double click **setup.exe** in the directory of **MSDE2000\MSDE2000A** in the Install Disk to begin to install MSDE2000A.


#### 2. Start Up Server

After the software has been installed successfully, the user can start up MSDE by one of the following ways:

- (1) Restart the PC machine;
- (2) Click **Start à Program à Startup** to pop up the SQL Server window. In the SQL Server window, click  to start up the SQL Server.

After starting up MSDE successfully, the SQL Server icon  will be displayed in the bottom right corner of the computer desktop. The green triangle indicates the startup situation.

### 3.2.2 Install the Software Protection Driver

Double click Software Protection Driver\SPI730\setup.exe  to start to install; several seconds later, the following figure is displayed. Click **Next** key to continue.



When the following figure pops up, select **Complete** item and click **Next** key.



In the following figure, click **Install** key to pop up **Windows XP/2003 detected** interface.



In the **Windows XP/2003 detected** interface, as following figure shows, click **Yes** key to start to install.





After finish installing, the following figure is displayed, and then click **Finish** key to confirm.



### 3.2.3 Install Smart ECG-Viewer




Double click SmartECGViewer\setup.exe to display the following figure.




In the above figure, click **Next** key to pop up the following License Agreement interface, select **I**

accept the terms in the license agreement item and click **Next** key.

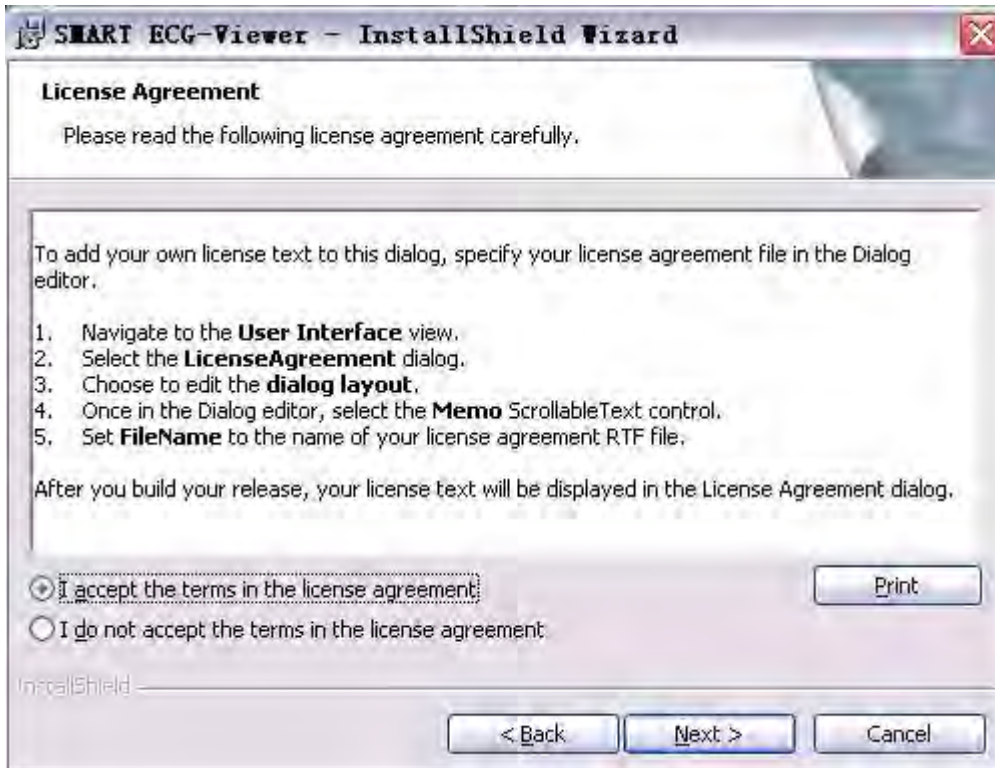

When the following figure is displayed, enter the User Name and Organization, select **Anyone who uses this computer (all users)** item, and then click **Next** key.

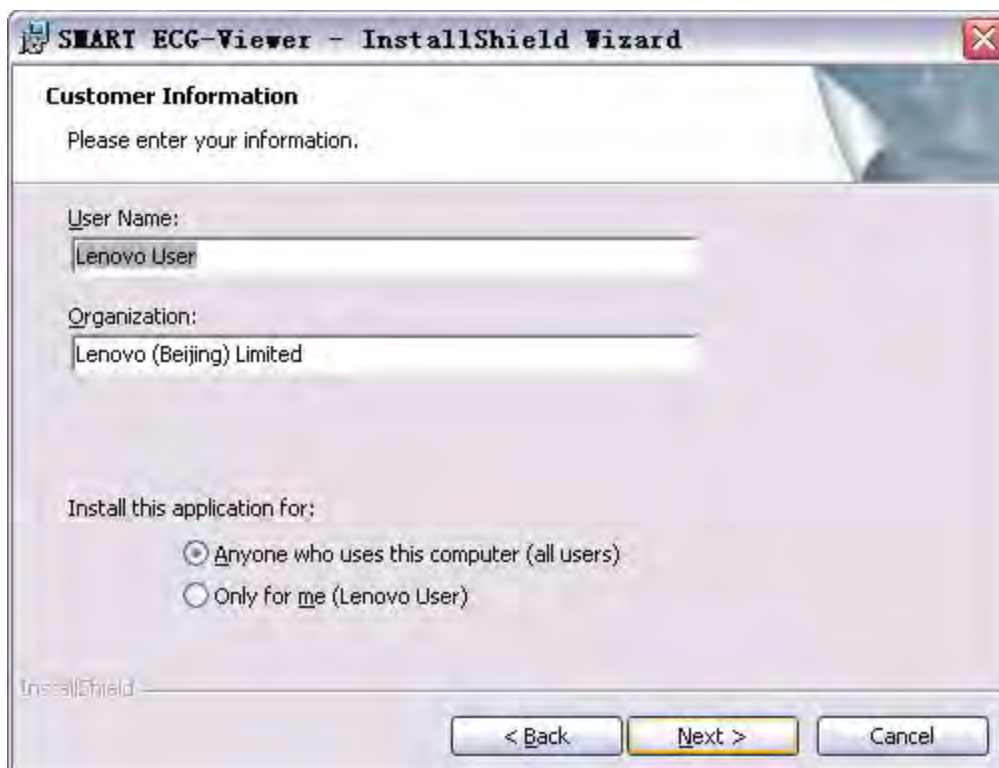

In the following figure, the user can click **Next** key to install the software under a default directory in the system disk, such as C:\Program Files\SmartECGViewer.


The user can also define the installation path by himself. Click **Change** key to pop up the Change



Current Destination Folder interface.



In the Change Current Destination Folder interface, select a new installation position, and click **OK** key to return to the above interface. Then click **Next** key to continue.

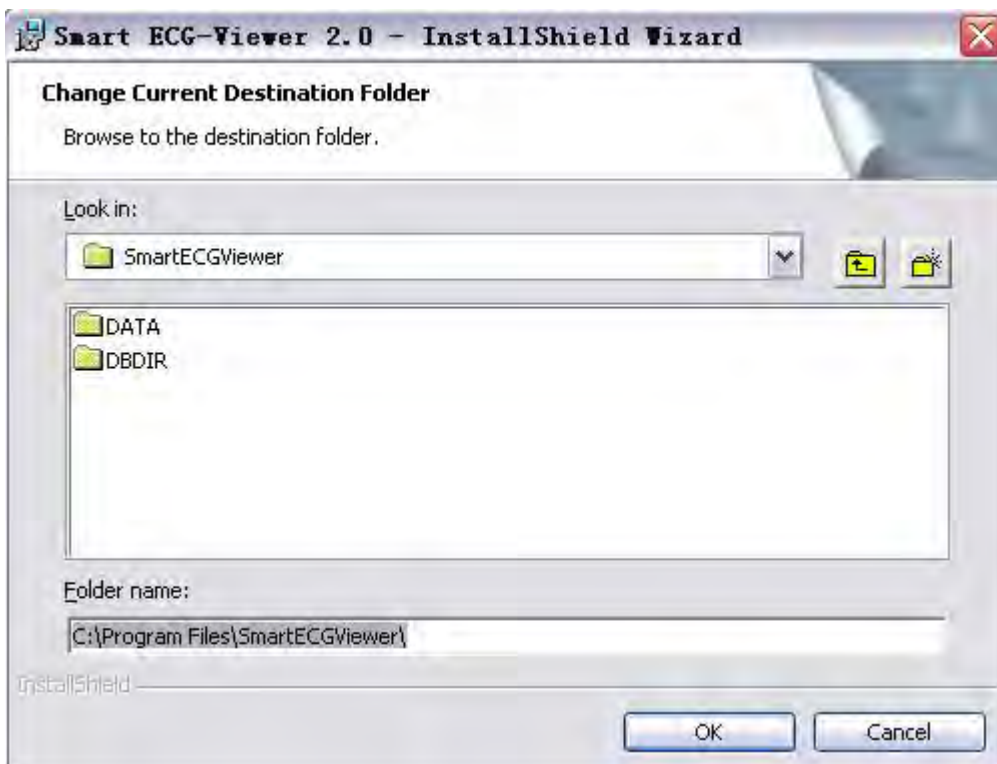

When the following figure pops up, click **Install** key to start to install Smart ECG-viewer.



After finish installing, the following figure is displayed, and then click **Finish** key to confirm.



### 3.2.4 Install PDF creator


Double click PDFCreator\PDFCreator.msi  in the Install Disk to display the following figure. Select **I accept the terms in the License Agreement** item and click **Next** key to start to install.



The following figure will be displayed in the installing process.



After finish installing, the following figure is displayed, and then click **Finish** key to confirm.

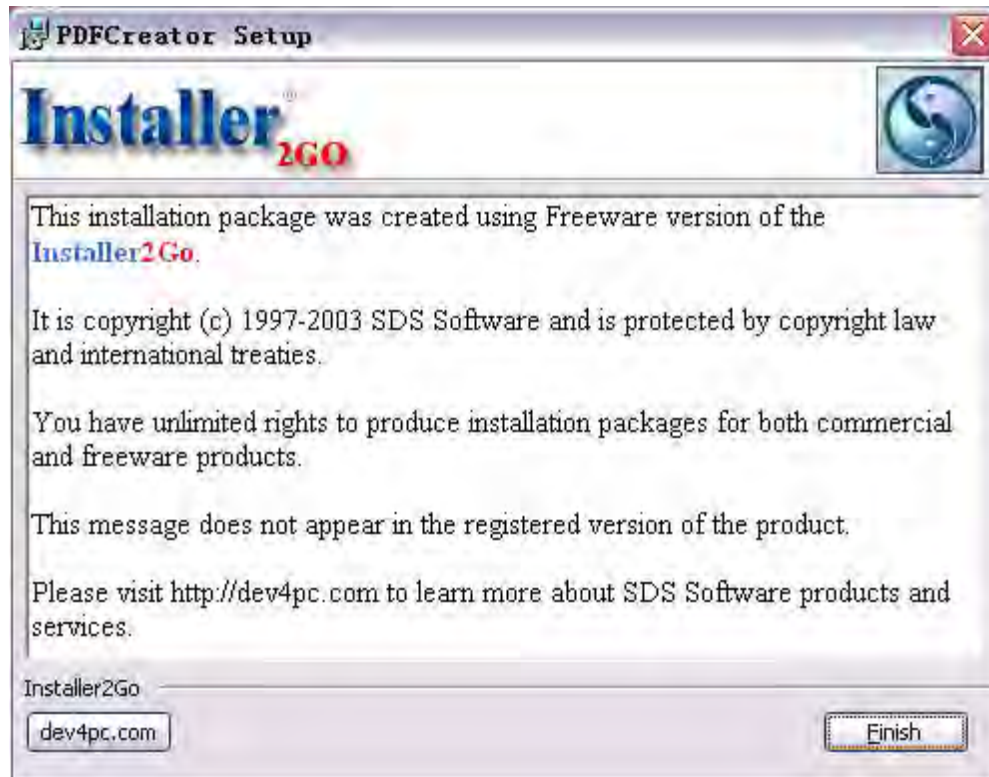

## 4 Operation

### 4.1 Startup

Start this software by the **shortcut on the desktop** or the **shortcut in the program menu**.




### 4.2 Log In



The initial user name is su and initial password is su. The software can remember the user name of the latest logging in. If the user selects “**Remember**” item, when he logs in next time, he can click **OK** to enter directly after he inputs user name without inputting the password.

### 4.3 Import Data from Harddisk

Click **【Operation】** à **【Import Data】** in the main menu bar of the Patient Records window to pop up the following dialog box.

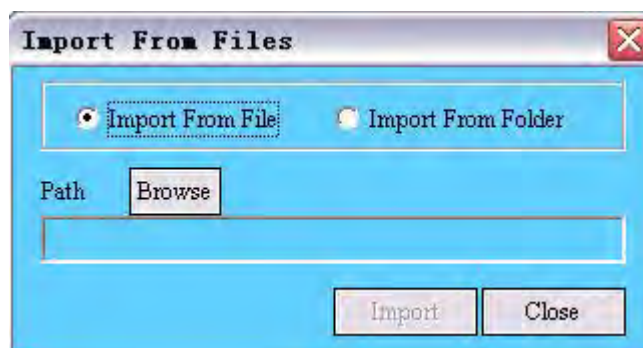

The user can import single file or data folder from harddisk into the system database.

Select **Import From File** or **Import From Folder**, and click **Browse** to appoint the path. Then the




user can click **Import** to import data to the system database.

**⚠WARNING⚠:**

After importing data into the database, the user should not modify the data save path. Otherwise error may happen when reading the data.

## 4.4 Receive ECG Data

Click **【Operation】** à **【Receive Data】** in the main menu bar of the Patient Records window to pop up the **Receive ECG Data** window.

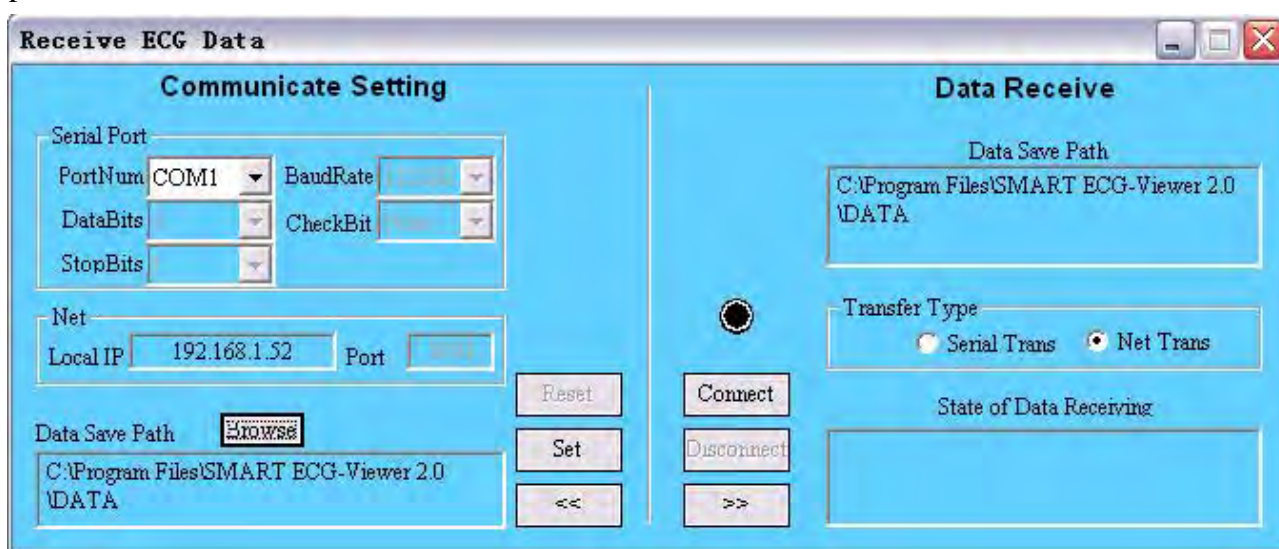

The user can receive ECG data from electrocardiograph through Serial port or Ethernet.

The following operation method is interpreted for the 3-channel electrocardiograph with 320×240 dot single color LCD screen.

For the 3-channel electrocardiograph with 192×64 dot single color LCD screen, please press **MENU** key on the control panel to enter the menu setting interface to set the corresponding menu items; For the 6-channel/12-channel electrocardiograph, please enter the Transmission Setup interface in the system setup window to set the corresponding menu items;

**When receiving ECG data through Serial port, the operation method is as follows:**

- 1) Connect the serial port of PC machine and RS232 socket of electrocardiograph with serial cable recommended by EDAN;
- 2) Set the **Transfer Type** as **Serial Trans** in the above Receive ECG Data window; Click **Reset** button to set the right **PortNum** and click **Set** button to confirm; Then click **Connect** button to link the electrocardiograph to PC machine;
- 3) When the user wants to transfer ECG data automatically, firstly the **AUTO**

**TRANSFER** in **GENERAL** interface of electrocardiograph should be set as **UART AUTO**. Under **AUTO** mode or **OFF** mode, ECG data can be transferred through serial port to PC machine automatically after an ECG recording is finished.

- 4) When the user wants to transfer ECG data in Recall Window of electrocardiograph to PC machine, firstly **AUTO TRANSFER** in **GENERAL** interface of electrocardiograph should be set as **UART AUTO**. Then press **RECALL** key of electrocardiograph to enter the Recall window. To transfer ECG data one by one, please select ECG data in the Recall window and click **TRANSMIT** button, and then the selected data will be transferred to PC machine through serial port. To transfer all ECG data in the Recall window, click **TRANS ALL** button and all the ECG data will be transferred to PC machine through serial port.
- 5) When the user wants to transfer ECG data manually, firstly record an ECG data under **AUTO** mode or **OFF** mode, and then enter the **GENERAL** interface of electrocardiograph. Press **Left Arrow/Right Arrow** key of electrocardiograph to set the **UART TRANSFER** as **Transferring**, which means the ECG data recorded just now is being transferred through serial port. If transferring is over, the **UART TRANSFER** item returns to “stop”.

**When receiving ECG data through Net, the operation method is as follows:**

- 1) Connect the net interface of PC machine and the net interface of electrocardiograph with Ethernet cable recommended by EDAN;
- 2) Set the **Transfer Type** as **Net Trans** in the above Receive ECG Data window; Click **Connect** button to link the electrocardiograph to PC machine;
- 3) Set the **REMOTE IP** in the **SYSTEM** interface of electrocardiograph same as the **IP address** of PC machine; Set the first three numbers of **LOCAL IP** in the **SYSTEM** interface of electrocardiograph same as the first three numbers of the **Default gateway** of PC machine, and the last number of **LOCAL IP** can be set at discretion.

Take the following figure--**Internet Protocol (TCP/IP) Properties**-- for example:

Firstly, the user can follow these operation steps to display the **Internet Protocol (TCP/IP) Properties** interface in PC machine:


Right click **My Network Places** and select **Properties** to display **Local Area**

**Connection** → Right click **Local Area Connection** and select **Properties** to display the **Local Area Connection Properties** interface → Double click **Internet Protocol (TCP/IP)** in **General** to pop up the **Internet Protocol (TCP/IP) Properties** interface.

Set the **REMOTE IP** in electrocardiograph same as the **IP address** in the following interface, and that is 192.168.1.143; Set the first three numbers of **LOCAL IP** in electrocardiograph same as the first three numbers of **Default gateway** in the following interface, and that is 192.168.1.\*\*\*; The last number of **LOCAL IP** in electrocardiograph can be set at discretion.

**Note:** The first three numbers of both IP address and Default gateway in PC machine must be set the same.

**Note:** Local IP can not collide with Remote IP or other net device IP.

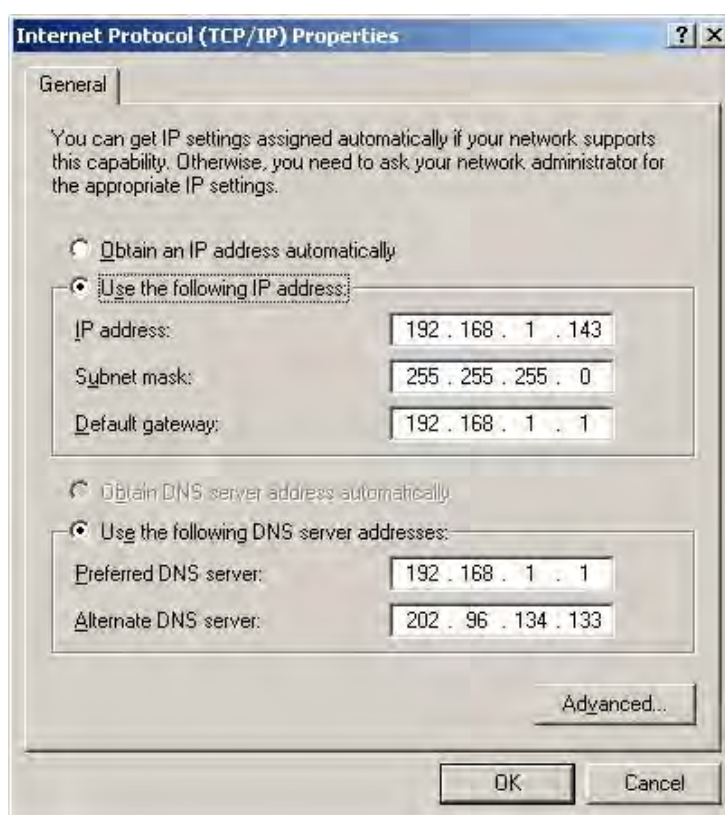

- 4) When the user wants to transfer ECG data automatically, firstly the **AUTO TRANSFER** in **GENERAL** interface of electrocardiograph should be set as **Net AUTO**. Under **AUTO** mode or **OFF** mode, ECG data can be transferred through net to PC machine automatically after an ECG recording is finished.
- 5) When the user wants to transfer ECG data in Recall Window of electrocardiograph to PC machine, firstly the **AUTO TRANSFER** in **GENERAL** interface of



electrocardiograph should be set as **Net AUTO**. Then press **RECALL** key in electrocardiograph to enter the Recall window. To transfer ECG data one by one, please select ECG data in the Recall window and click **TRANSMIT** button, and then the selected data will be transferred to PC machine through net. To transfer all ECG data in the Recall window, click **TRANS ALL** button and all the ECG data will be transferred to PC machine through net.

- 6) When the user wants to transfer ECG data manually, firstly record an ECG data under **AUTO** mode or **OFF** mode, and then enter the **GENERAL** interface of electrocardiograph. Press **Left Arrow/Right Arrow** key of electrocardiograph to set the **NET TRANSFER** as **Transferring**, which means the ECG data recorded just now is being transferred through net. If transferring is over, **NET TRANSFER** item returns to “stop”.

The user can appoint the new data save path in the Receive ECG Data window. Click **Browse** to pop up the Path Selection dialog box, and then the user can appoint the new data save path.

**Note:** The quantity of data to be transmitted through Serial Port is huge, and transmission is limited by the Serial Port transmission protocol, both of which result in the low speed of transmitting. So transmission of a great number of data through Serial Port is not recommended.

## 4.5 View and Edit Patient and Record Information

After log in, the software will enter the Patient records window automatically, as figure 4-1 shows. In this window, the user can view, modify, delete, and can also assign case records. Besides, the user can delete several patient records concurrently, delete several case records concurrently, or assign several case records concurrently.

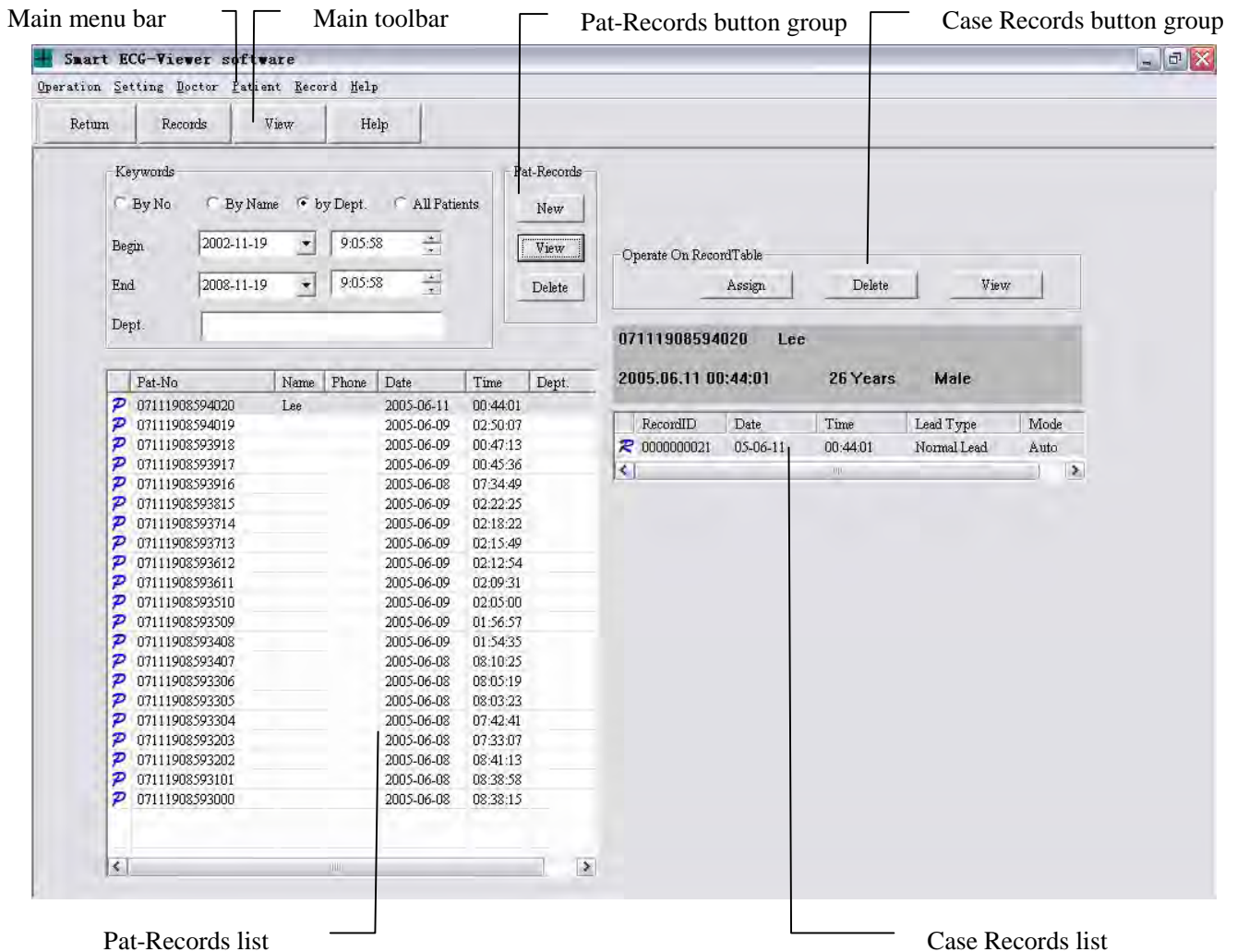


Figure 4-1 Patient records window

### 4.5.1 New Patient Record

Click **New** in the Pat-Records button group or click **【Patient】** à **【New】** in the main menu bar in Figure 4-1 to pop up the Patient Infos window. The user can build new patient record in this window. And the No. will be given by the system automatically.

Figure 4-2 Patient Infos window

**Note:** Please refer to Chapter 4.15 “User Defined Setting” for detailed information on the setting method of User Defined.

In the Patient Infos window, the user can select doctor from the Doctor pull-down list. If there is no record of the doctor you want to choose, you can click **New** button to pop up the Doctor Records window to add this doctor’s record. The Doctor Records window is as follows.

Name	Phone	Dept.	Hospital
Lee	156665464656	ECG	SUN Hospital

Figure 4-3 Doctor Records window

To add a doctor record, the user can input the doctor's information in the Information Input field, and click **Add**. Then this doctor record will be added in the Doctor-Records list.

To delete a doctor record, the user can select the doctor record to be deleted in the Doctor-Records list, and click **Delete**. Then this doctor record will disappear in the Doctor-Records list.

To modify a doctor record, the user can select the doctor record to be modified in the Doctor-Records list, and the information of the doctor record will be loaded to the Information Input field automatically, and then the user can modify the information. After finishing modifying, click **Modify** to save it.

After finishing operation, the user can click **Close** to exit from the Doctor Records window.

## 4.5.2 View and Modify Patient Record

After select one record in the pat-records list in Figure 4-1, the user can click **View** in the Pat-Records button group or click **【Patient】→【View】** in the main menu bar to pop up Patient Infos window to view the patient basic information selected. In this window, the patient basic information selected can be modified or saved.

The screenshot shows a window titled "Patient Infos" with a close button in the top right corner. The window is divided into two main sections: "Patient Basic Info" and "Diagnose".

**Patient Basic Info:**

- No: 07112117022300
- Name: Lee
- Age: 35 Years
- Gender:  Male  Female
- Race: Black
- Phone: [Empty]
- Height(cm): 172
- Dept.: [Empty]
- Weight(kg): 69
- Ward: [Empty]
- BP(mmHg): [Empty] / [Empty]
- Drug: [Empty]
- Time: 08/06/08 13:39
- Address: [Empty]
- Technician: [Empty]
- Disease History: [Empty]

**Diagnose:**

- Doctor: Jakie (dropdown menu)
- Phone: [Empty]
- Dept.: [Empty]
- Hospital: Sun Hospital
- [Empty text area]

**User Defined:**

- Clinic ID: B0256824
- UserDefine: content1
- UserDefine: content2
- [Empty text area]

At the bottom of the window, there are three buttons: "Modify", "Insert", and "Close".

## 4.5.3 Delete Patient Record

### 1. Delete Single Record


Select the patient record to be deleted in the pat-records list in Figure 4-1, and click **Delete** in the Pat-Records button group or click **【Patient】 à 【Delete】** in the main menu bar to delete this patient record. At the same time, all the relative case records and data of that patient will be deleted. Before deleting, the system will give the prompt dialog box to remind the user to confirm.

### 2. Delete Multi-Records

Click the first patient record to be deleted in the patient records list in Figure 4-1, and press the **Shift** key on the keyboard, and then click the last patient record to be deleted in the patient records list in Figure 4-1; The user can also press **Ctrl** key on the keyboard and then select the patient record one by one. After select all the records to be deleted, click **Delete** in the Pat-Records button group or click **【Patient】 à 【Delete】** in the main menu bar to delete all the patient records selected. At the same time, all the relative case records and data of these patients will be deleted. Before deleting, the system will give the prompt dialog box to remind the user to confirm.

## 4.5.4 Assign Patient Record

Select one of the case records in the Case Records list in Figure 4-1, and click **Assign** button in the Case Records button group or click **【Record】 à 【Assign】** in the main menu bar to pop up confirming dialog box. After the confirming operation, the following **Assign** dialog box is displayed. Input patient ID in the **Assign** dialog box and click **OK** to assign the case record selected to this patient.

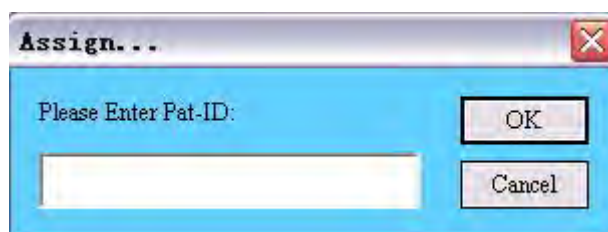

The software supports multi-case records assigning. The operation method of selecting several case records concurrently is similar to that of selecting several patient records concurrently, which is described in Chapter 4.5.3.

## 4.5.5 Delete Case Record

The operation method of deleting case record is similar to that of deleting patient record, which is described in Chapter 4.5.3.

## 4.5.6 View Case Record

For the case records of Auto mode, click **View** button in the Case Records button group or click **【Record】 à 【View】** in the main menu bar to switch to the ECG curve view window, as figure 4-5 shows.

For the case records of Rhythm1/Rhythm3 mode, click **View** button in the Case Records button group or click **【Record】 à 【View】** in the main menu bar to switch to the Print Interface (Rhythm Mode), as figure 4-10 shows.

## 4.5.7 Search Patient Record

The patient record supports fuzzy search.

When the Search Mode is set as **By No.**, within the date and time range, all the patient records whose Numbers begin with the character string in the **No. input field** will be listed in the Patient Records window.


When the Search Mode is set as **By Name**, within the date and time range, all the patient records whose name begin with the character string in the **Name input field** will be listed in the Patient Records window.

When the Search Mode is set as **By Dept.**, within the date and time range, all the patient records whose name begin with the character string in the **Dept. input field** will be listed in the Patient Records window.

When the Search Mode is set as **All Patients**, all the patient records will be listed in the Patient Records window.

After the search condition changes, the system will update the search result automatically.



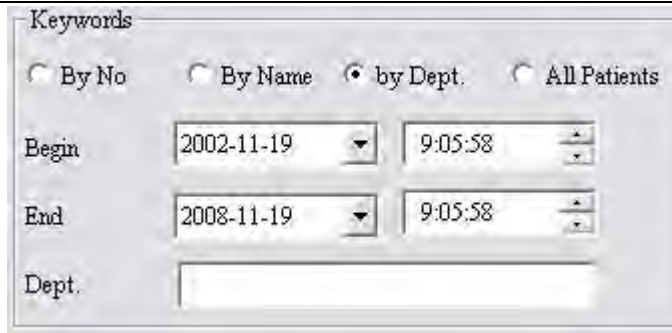

## 4.6 Case Records Window

Click **Records** key in the main toolbar of Figure 4-1 to switch to the Case Records window, as the figure 4-4 shows. In this window the user can search, view, delete or assign case records. The operation method of viewing, deleting and assigning case records is similar to the corresponding operations in Patient Records window.

**Note:** Only when the user clicks **Search** key after input the search condition, can the search result be updated in the record table.

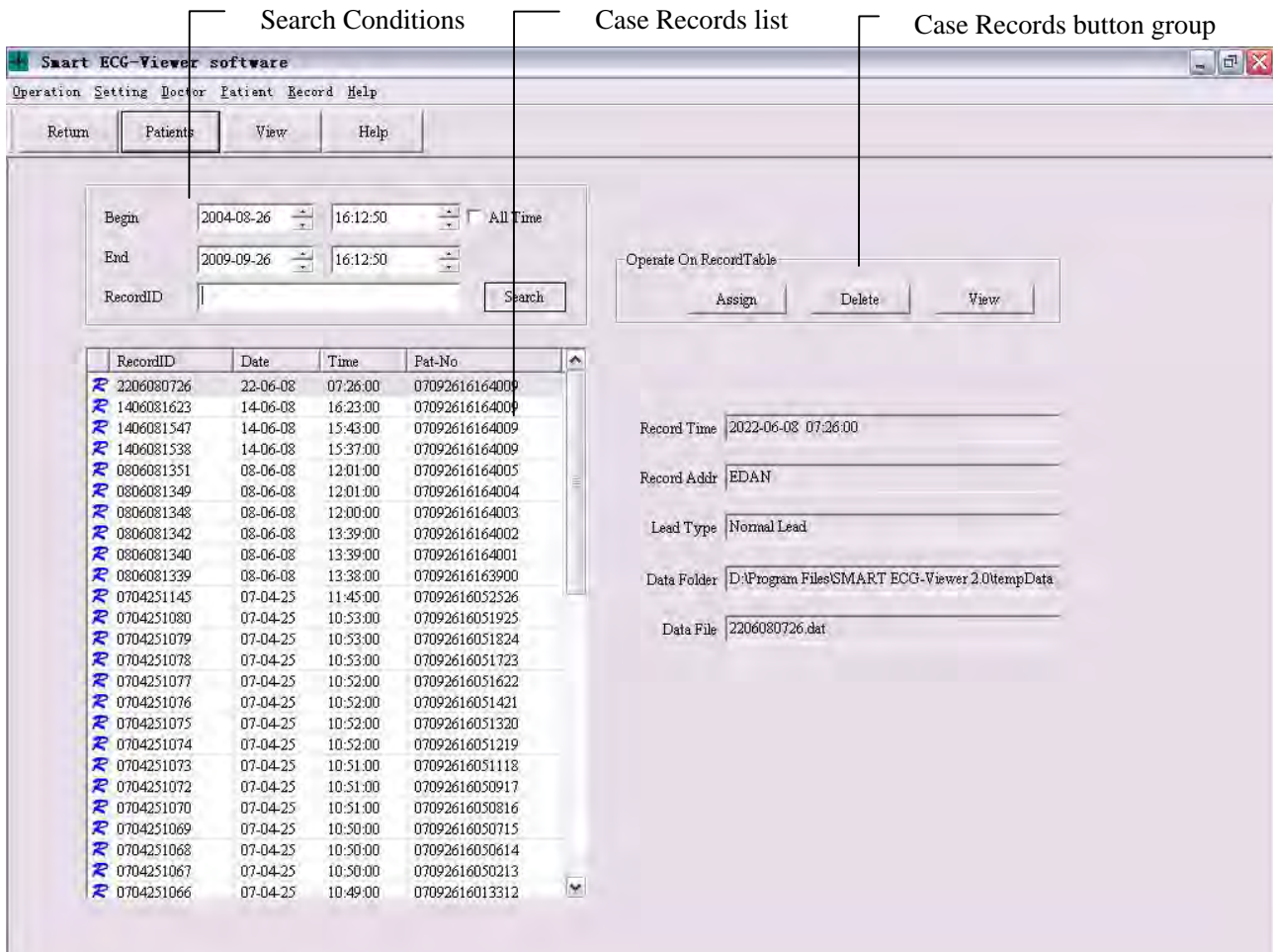



Figure 4-4 Case Records window

## 4.7 View ECG Curve

For the case records of Auto mode, the user can switch to the Mean interface of ECG curve view window by one of the following ways:

For the case records of Rhythm1/Rhythm3 mode, the user can switch to the Print Interface (Rhythm Mode) by one of the following ways:

- Double-click patient records list or case records list in the Patient Records window;
- Double-click case records list in the Case Records window;
- Click **View** button in the case records button group;
- Click **View** button in the main toolbar;

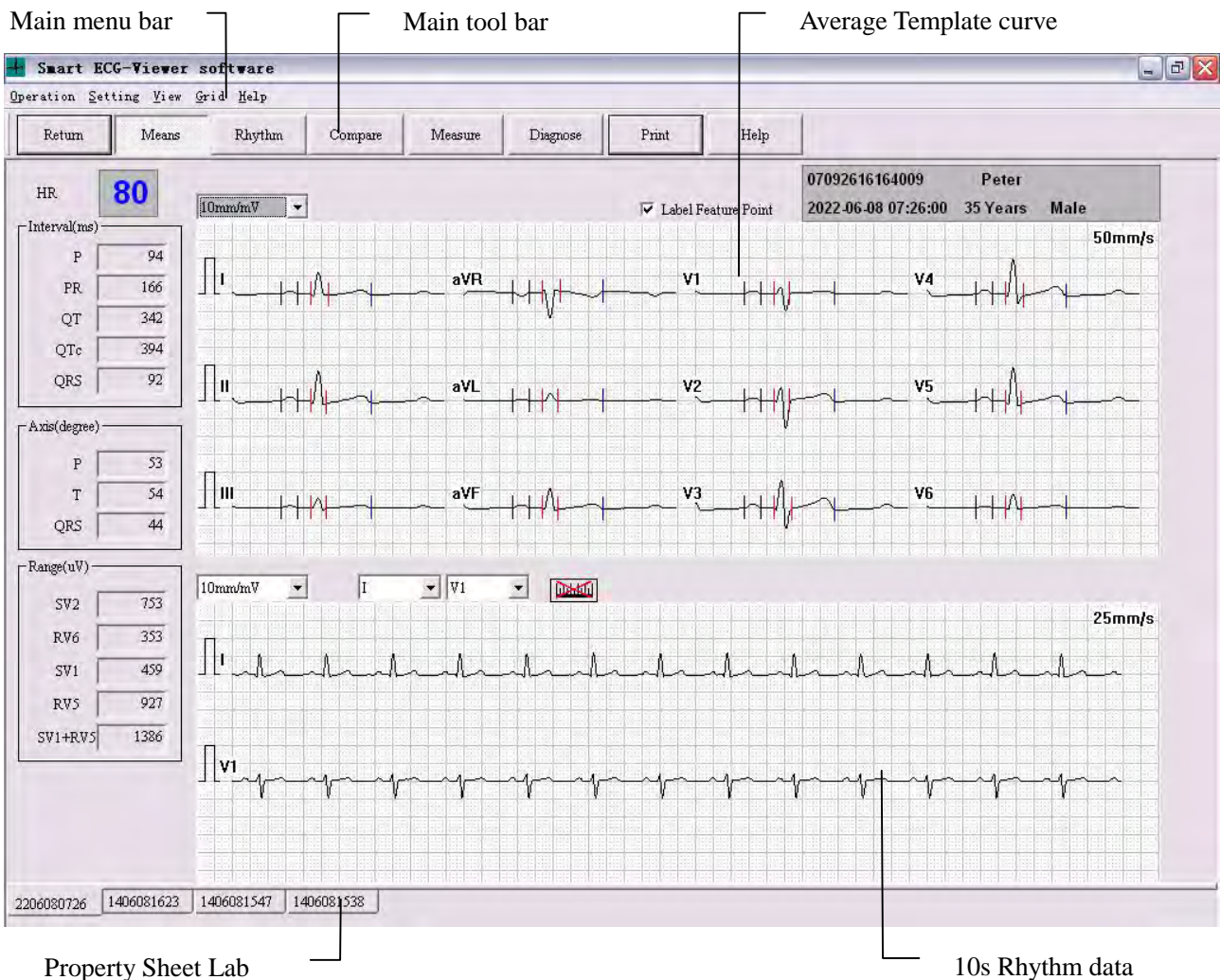

**Figure 4-5 Mean interface of ECG curve view window (Select Label Feature Point)**

In the Mean interface of ECG curve view window, the user can view the patient's 12-leads ECG Average template curve, parameter measuring result (including HR, Intervals, Axes and Range etc.). The user can zoom in those Average templates by clicking the Average template curve to be viewed,



as the figure 4-6 shows.

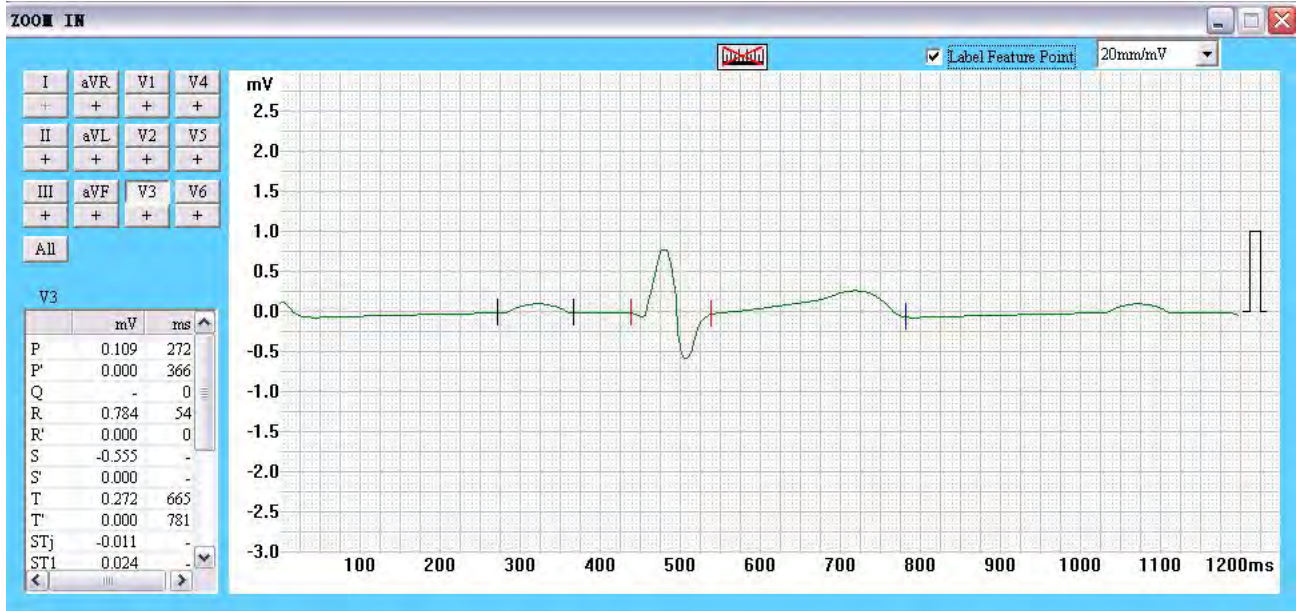


Figure 4-6 amplificatory Average Template interface (Select Label Feature Point)

In the Average template, the start-point and end-point of P wave, the start-point and end-point of QRS wave and the end-point of T wave can be displayed. And it supports the adjustment of gain and manual measurement etc.

In the Average template, the user can also select two leads to view the rhythm data for 10s, or to measure the axes manually, or to change the gain. By clicking the Property Sheet Label on the bottom, the user can switch between several case records.

In Figure 4-5 and Figure 4-6, the user selects **Label Feature Point**, and the curve feature points are marked; If the user does not select **Label Feature Point**, the curve feature points will not be marked, as Figure 4-7 and Figure 4-8 shows.

# Smart ECG-Viewer Data Management Software User Manual

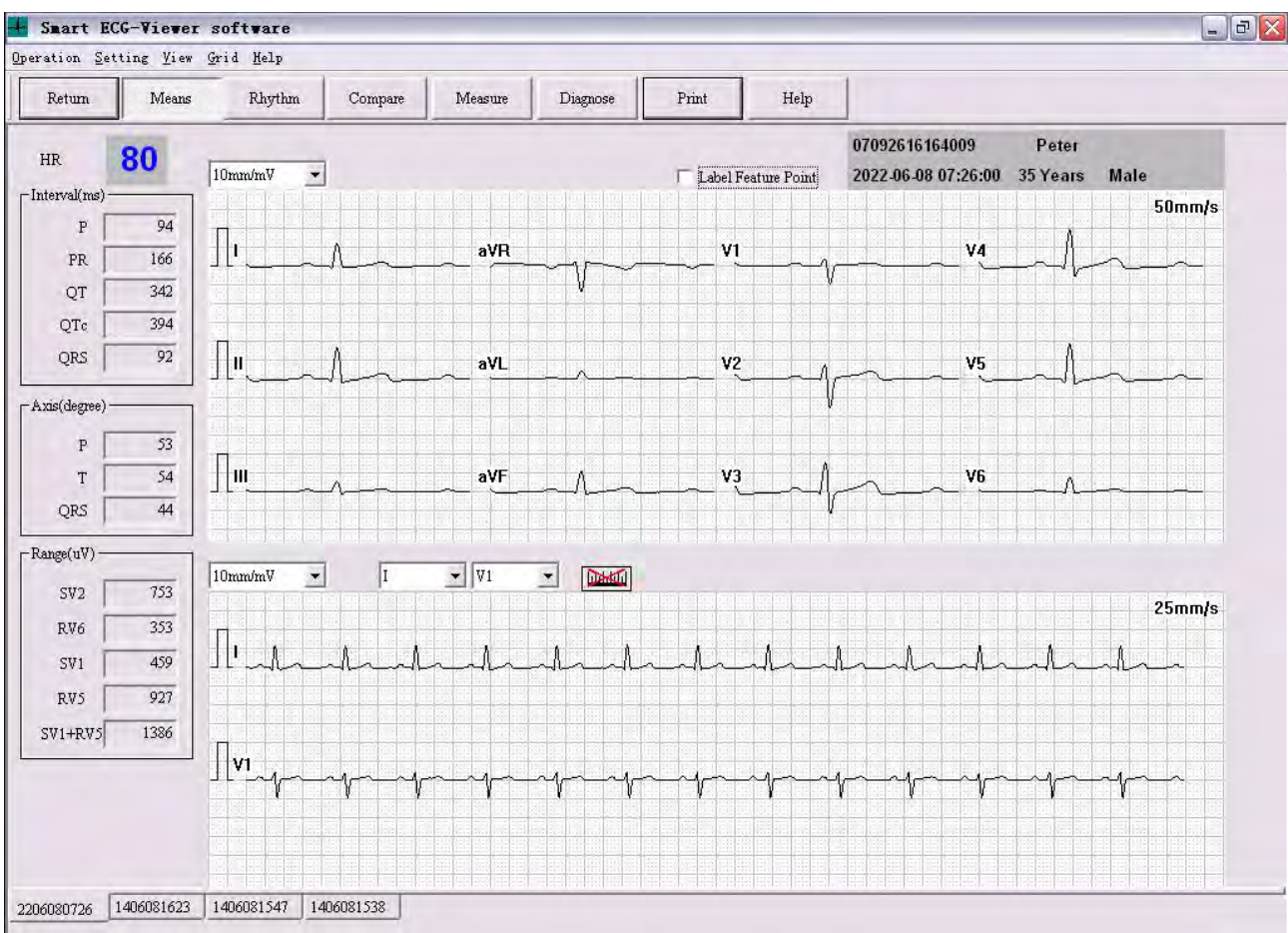


Figure 4-7 Mean interface of ECG curve view window (Do not Select Label Feature Point)

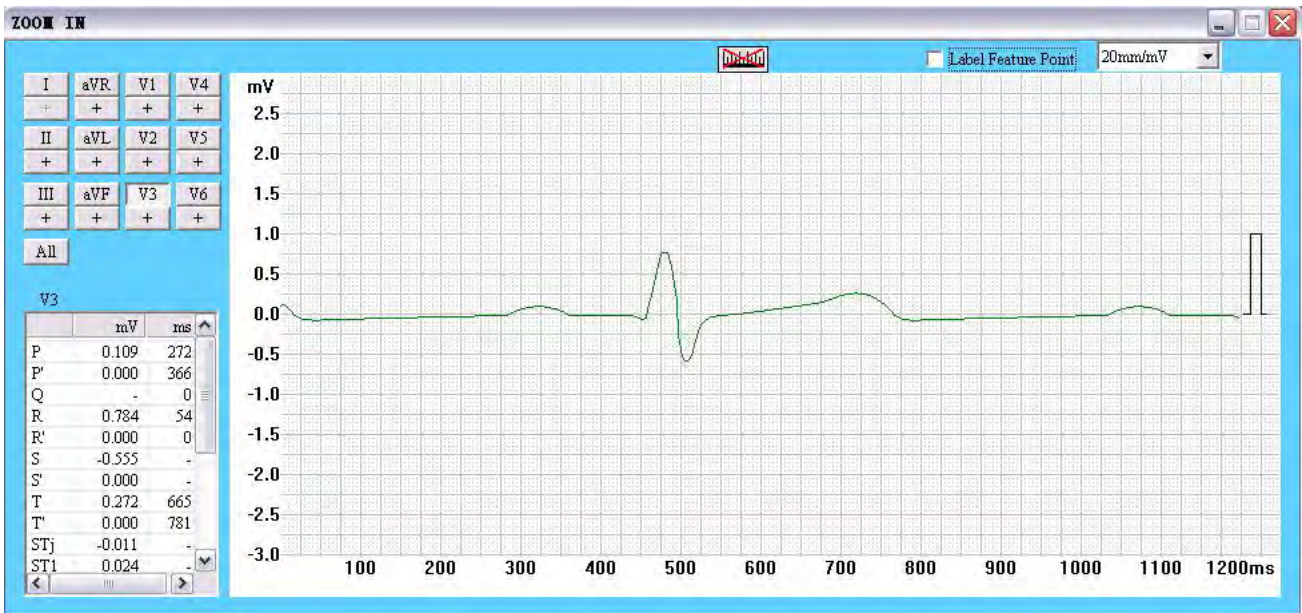



Figure 4-8 amplification Average Template interface (Do not Select Label Feature Point)



## 4.8 Rhythm Data Browse


Click **Rhythm** button in the main toolbar of Figure 4-5 to switch to the browse interface of 10s rhythm data and curve. In the interface, the system supports 3-leads per screen display, 6-leads per screen display and 12-leads per screen display. And the user can also set the gain and paper speed in this interface. Besides, the scrolling display of lead and moving display of time axes can be carried out by the **Up/Down/Left/Right** key on the top.

**3-leads per screen display:**

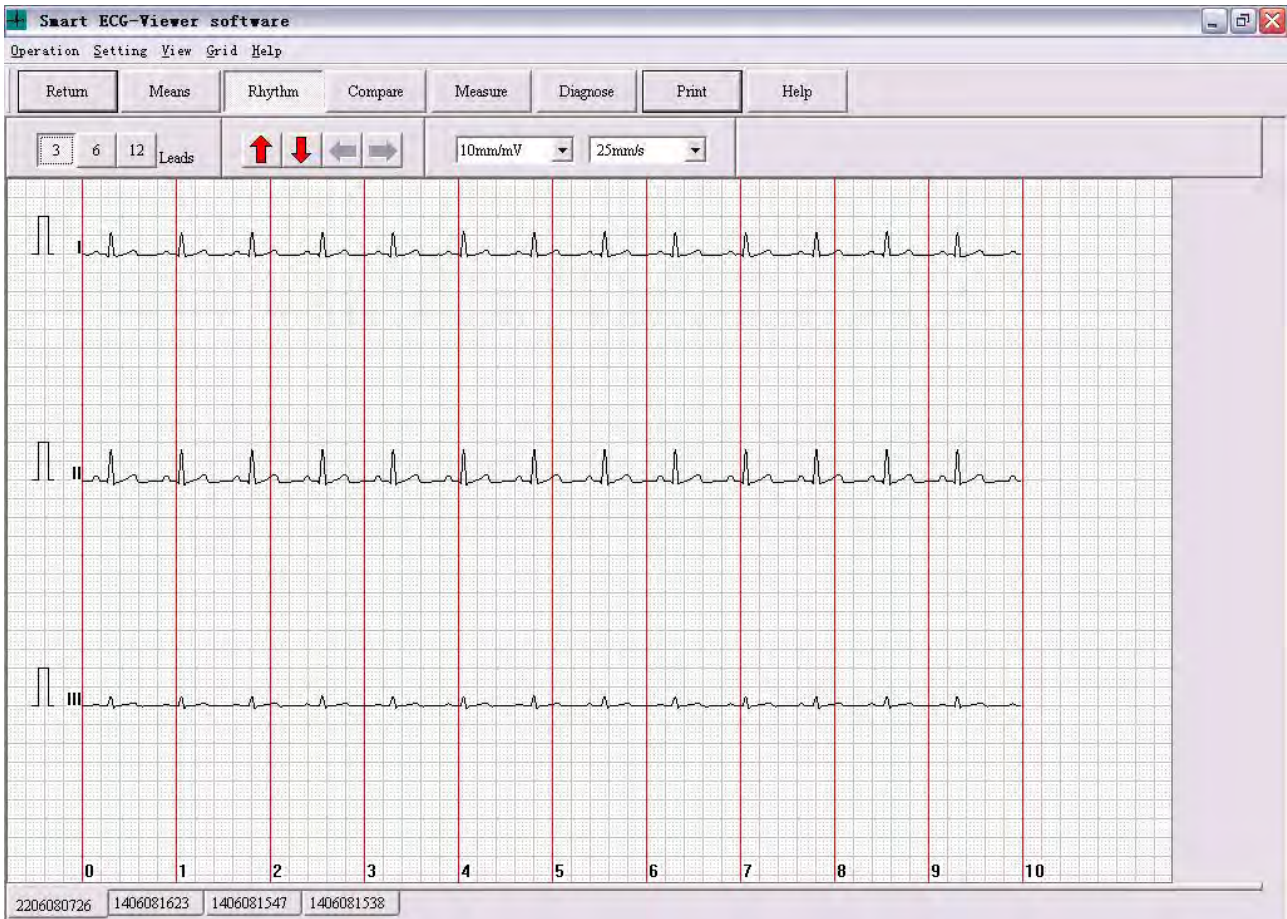




6-leads per screen display:

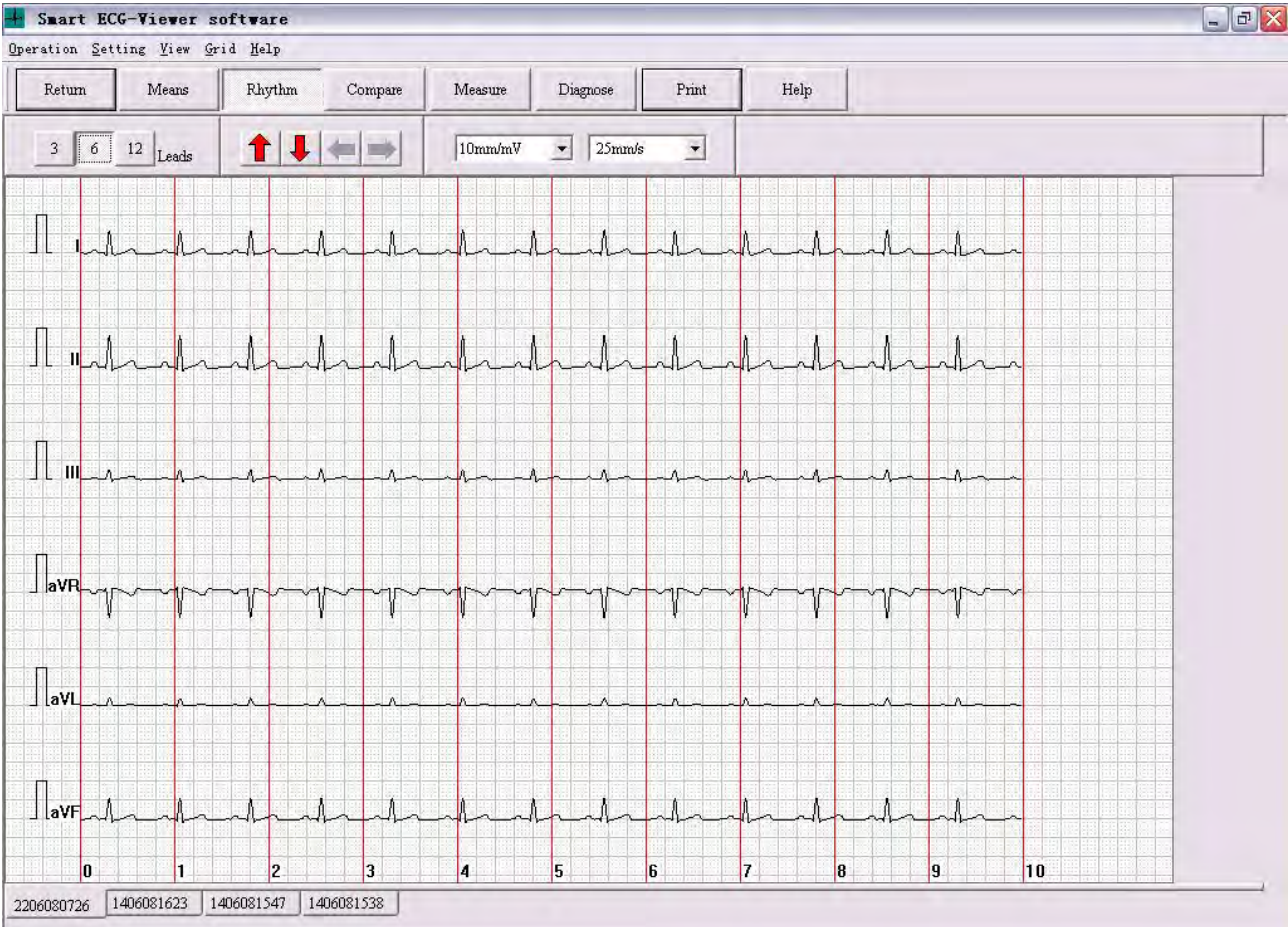

12-leads per screen display:

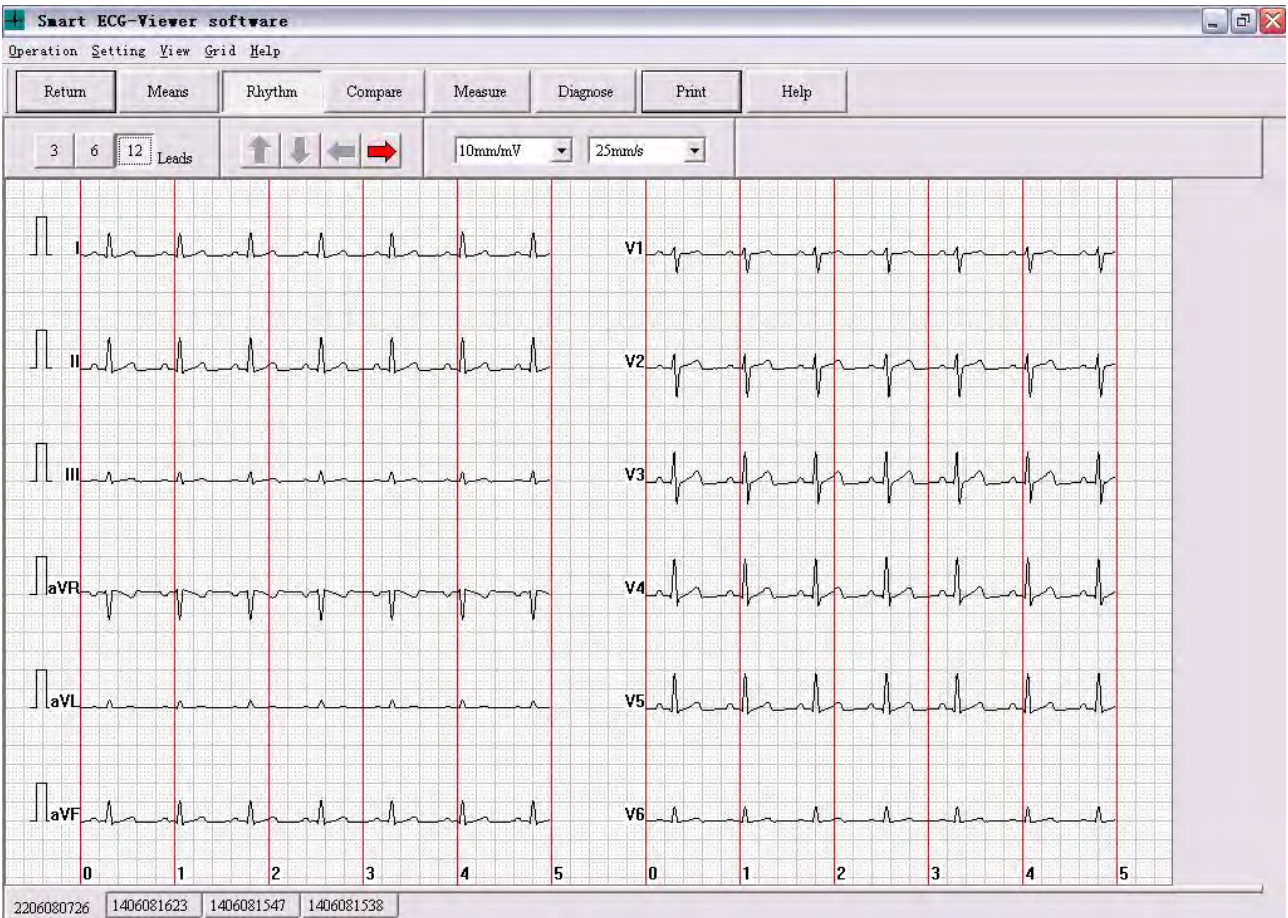





## 4.9 Multi-Cases Rhythm Data Comparing Browse

Click **Compare** button in the main toolbar of Figure 4-5 to switch to the comparing browse interface of multi-cases. In this interface, the user can view 10s rhythm data of every lead of multi-cases, and mean data of every lead, and auto-diagnosing results.

**Average template of multi-cases comparing browse:**

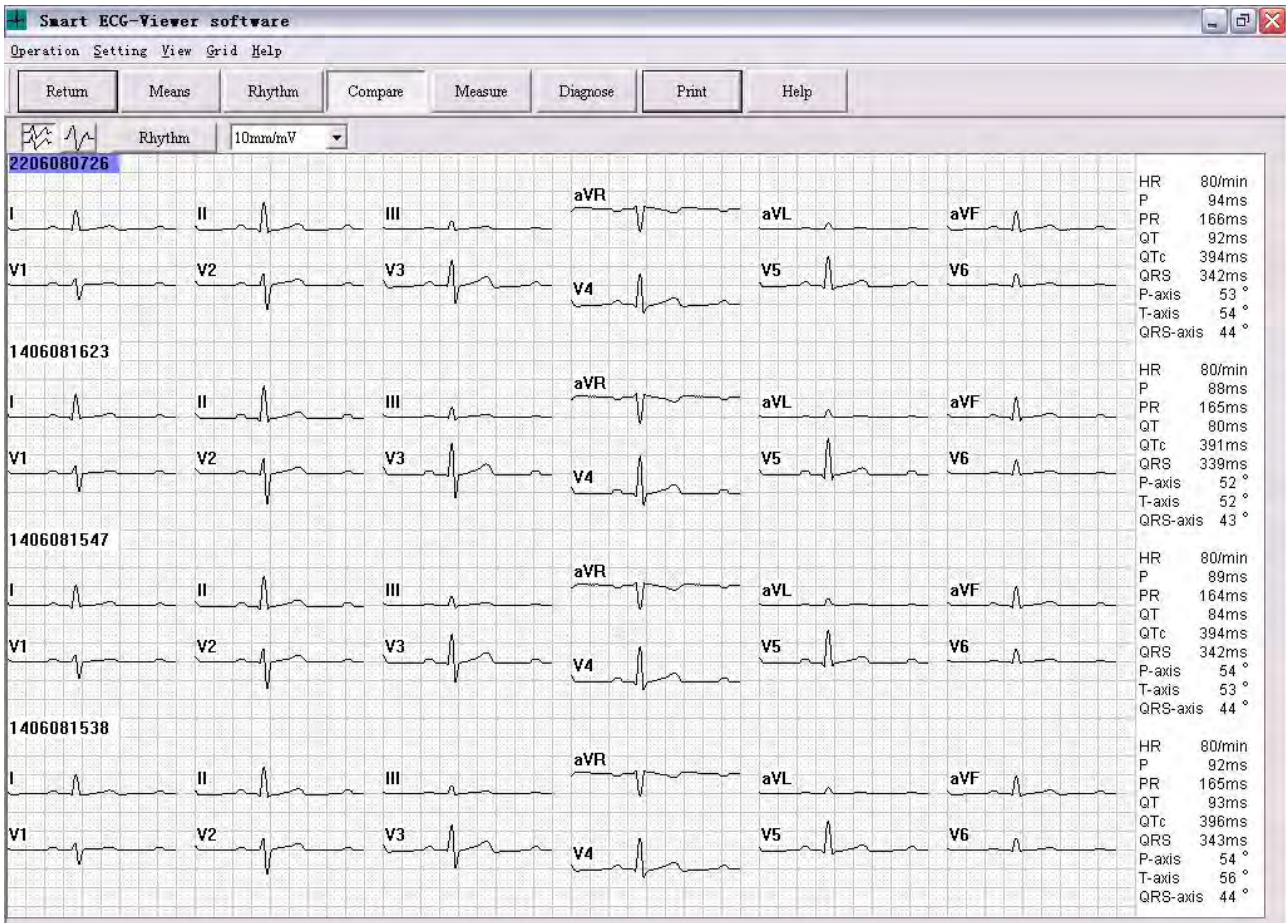

10s rhythm of multi-cases comparing browse:

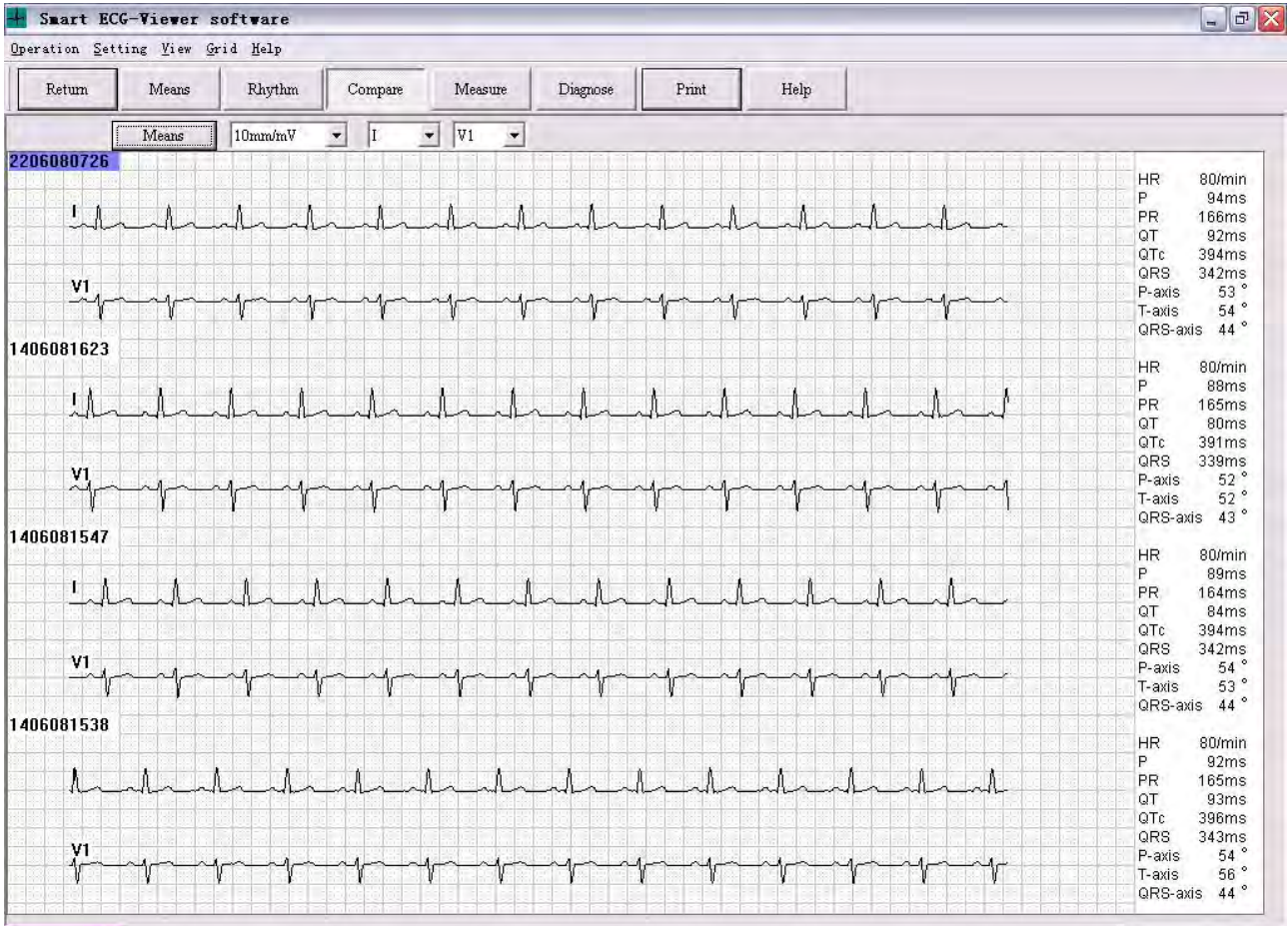




## 4.10 Auto Measurement Parameter View

Click **Measure** button in the main toolbar of Figure 4-5 to pop up the dialog box of ECG Auto Measurement Result, as the following figure shows.

	I	II	III	aVR	aVL	aVF	V1	V2	V3	V4	V5	V6
Type	qRs	Rs	rs	Qr	r	Rs	rs	rS	RS	Rs	Rs	rs
Pa1(mV)	0.066	0.114	0.045	-0.087	0.000	0.082	0.072	0.074	0.109	0.114	0.115	0.000
Pa2(mV)	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000
Ra1(mV)	0.584	0.827	0.240	0.064	0.171	0.535	0.173	0.361	0.784	0.971	0.927	0.353
Ra2(mV)	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000
Sa1(mV)	-0.037	-0.094	-0.067	0.000	0.000	-0.078	-0.459	-0.753	-0.555	-0.255	-0.123	-0.046
Sa2(mV)	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000
Ta1(mV)	0.096	0.154	0.055	-0.122	0.000	0.106	0.074	0.192	0.272	0.202	0.152	0.055
Ta2(mV)	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000
Fd(ms)	92	94	97	97	97	93	92	94	94	94	93	97
Qd(ms)	22	0	0	44	0	0	0	0	0	0	0	0
Rd1(ms)	50	65	60	36	89	64	47	50	54	60	64	67
Rd2(ms)	0	0	0	0	0	0	0	0	0	0	0	0
Td(ms)	114	119	118	114	117	119	116	115	116	116	116	119
PR(ms)	165	166	167	167	167	166	166	165	167	166	167	166
QT(ms)	342	343	342	341	341	344	341	343	342	342	343	344
QRS(ms)	438	439	439	439	439	438	439	438	439	438	439	438
VAT(ms)	12	40	38	40	0	41	37	38	39	40	40	40
STj(mV)	-0.034	-0.054	-0.016	0.040	-0.011	-0.035	-0.003	-0.016	-0.011	-0.043	-0.056	-0.032
ST1(mV)	-0.024	-0.032	0.000	0.027	-0.011	-0.016	0.018	0.047	0.024	-0.011	-0.027	-0.019
ST2(mV)	0.000	-0.003	0.008	-0.005	-0.011	0.002	0.031	0.063	0.060	0.026	0.003	-0.006
ST3(mV)	0.022	0.032	0.013	-0.029	-0.011	0.024	0.039	0.090	0.111	0.063	0.038	0.008
ST20(mV)	-0.032	-0.040	-0.005	0.032	-0.016	-0.022	0.015	0.034	0.013	-0.025	-0.037	-0.024
ST40(mV)	-0.018	-0.030	0.000	0.019	-0.013	-0.014	0.020	0.050	0.032	-0.006	-0.021	-0.016
ST60(mV)	-0.005	-0.013	0.003	0.000	-0.008	-0.006	0.026	0.061	0.050	0.016	-0.002	-0.011
ST80(mV)	0.014	0.013	0.013	-0.019	-0.005	0.013	0.034	0.074	0.082	0.042	0.016	0.000

## 4.11 Auto Diagnoses Result View

Click **diagnose** button in the main toolbar of Figure 4-5 to pop up the Diagnose dialog box. In this dialog box, the user can directly enter his own diagnosing opinions into the Diagnosis Information Input Field, or select the necessary information in the Auto Diagnose Tree and click **Add** key to add the selected information into the Diagnosis Information Input Field.

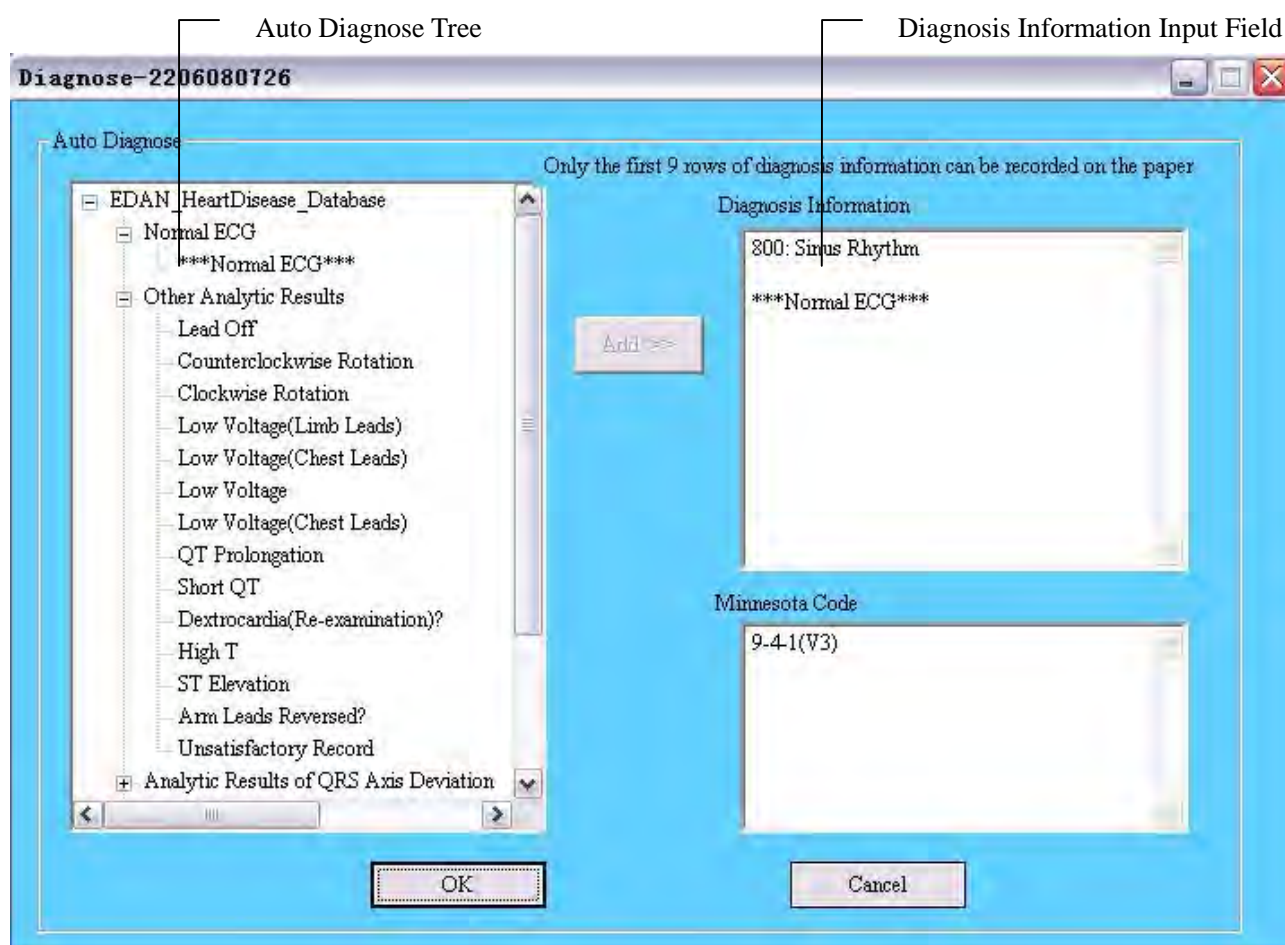

## 4.12 Preview and Print Diagnosing Report

For the case records of Auto mode, the user can preview and print report by one of the following ways:

- 【Operation】** à **【Print report】** in the main toolbar;
- Click **Print** key in the main tool bar of Figure 4-5;

The Print Interface (Auto Mode) will pop up, as Figure 4-9 shows.



Main menu bar

Main tool bar

The screenshot displays the 'Smart ECG-Viewer software' window. At the top is a 'Main menu bar' with options: Operation, Setting, Grid, Help. Below it is a 'Main tool bar' with buttons: Return, Format, Page Speed, Gain, RhythmLead, Sequence, Content, Preview, Print, Create PDF, Print Table. The main area contains patient information and ECG traces.

ID : 0806081340	2008 - 06 - 08 13 : 39	Ward NO :	Dept. :	Clinic ID : B0256824
Name : Sanming		HR :	80 bpm	Diagnosis Information :
Age : 35 yr	Gender: Male	P Dur :	90 ms	800: Sinus Rhythm
Height : 180 cm	Weight: 68 kg	PR int :	164 ms	***Normal ECG***
BP : mmHg	Race :	QRSInterp :	94 ms	
Medications :		QRS Dur :	341/393 ms	
Minnesota Code :	UserDefine :	QT/QTc axis :	54/44/54 °	
9-4-1(V3)	Content1	RV5/SV1 amp :	0.937/0.463 mV	
	UserDefine :	RV5+SV1 amp :	1.400 mV	
	Content2	RV6/SV2 amp :	0.367/0.772 mV	

Technician : Report Confirmed By :

ECG traces for leads: I, aVR, V1, V4, II, aVL, V2, V5, III, aVF, V3, V6, and a rhythm strip.

0.25 Hz ~ 25 Hz    25 mm/s    10 mm/mV    4 by 2.5 s + 1 rhythm    Sun Hospital    Smart ECG-Viewer V 2.0

Figure 4-9 Print Interface (Auto Mode)

For the case records of Rhythm1/Rhythm3 mode, the user can preview and print report by one of the following ways:

- Double-click patient records list or case records list in the Patient Records window;
- Double-click case records list in the Case Records window;
- Click **View** button in the case records button group;
- Click **View** button in the main toolbar;

The Print Interface (Rhythm Mode) will pop up, as Figure 4-10 shows.




**Figure 4-10 Print Interface (Rhythm Mode)**

In the Print Interface (Rhythm Mode), Format, Rhythm Lead, Sequence, Content and Print Table items can not be operated.

## 4.12.1 Display Format Setting

Click **Format** key in the Main tool bar of Print Interface in Figure 4-9 to pop up the Display Format Setting Interface.

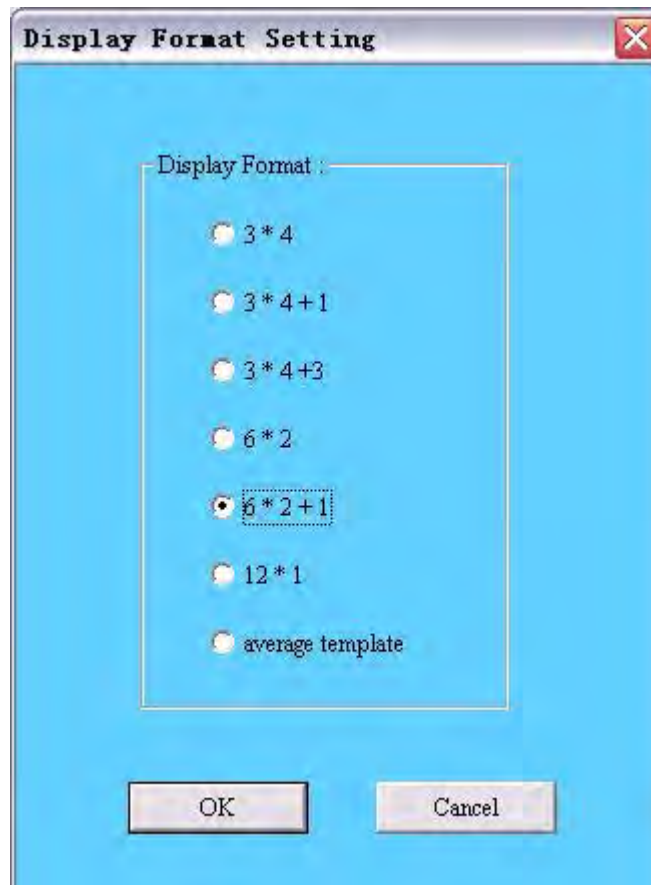

The user can select one of Display Formats in the above figure and click **OK** key. Then ECG wave in Print Interface will be displayed with the selected format. For example, when Display Format is set as 6\*2+1, the ECG wave in Print Interface will be like this:



## Smart ECG-Viewer Data Management Software User Manual

Smart ECG-Viewer software

Operation Setting Grid Help

Return	Format	Page Speed	Gain	RhythmLead	Sequence	Content	Preview	Print	Create PDF	Print Table
--------	--------	------------	------	------------	----------	---------	---------	-------	------------	-------------

ID : 0806081340	2008 - 06 - 08 13 : 39	Ward NO :	Dept. :	Clinic ID : B0256824
Name : Lee		HR : 80	bpm	Diagnosis Information :
Age : 35 yr	Gender: Male	P Dur : 90	ms	800: Sinus Rhythm
Height : 172 cm	Weight: 69 kg	PR int : 164	ms	
BP : mmHg	Race : Black	QRSInterp : 94	ms	***Normal ECG***
Medications :		QRS Dur : 341/393	ms	
Minnesota Code : 9-4-1(V3)	UserDefine : content1	QT/QTc axis : 54/44/54	°	
	UserDefine : content2	RV5/SV1 amp : 0.937/0.463	mV	
		RV5+SV1 amp : 1.400	mV	
		RV6/SV2 amp : 0.367/0.772	mV	

Technician :	Report Confirmed By :
--------------	-----------------------

I
II
III
aVR
aVL
aVF
II


V1
V2
V3
V4
V5
V6

0.25 Hz ~ 25 Hz    25 mm/s    10 mm/mV    2 by 5 s + 1 rhythm    Sun Hospital
Smart ECG-Viewer V 2.0

## 4.12.2 Page Speed Setting


Click **Page Speed** key in the Main tool bar of Print Interface in Figure 4-9 to pop up the Page Speed Setting Interface.

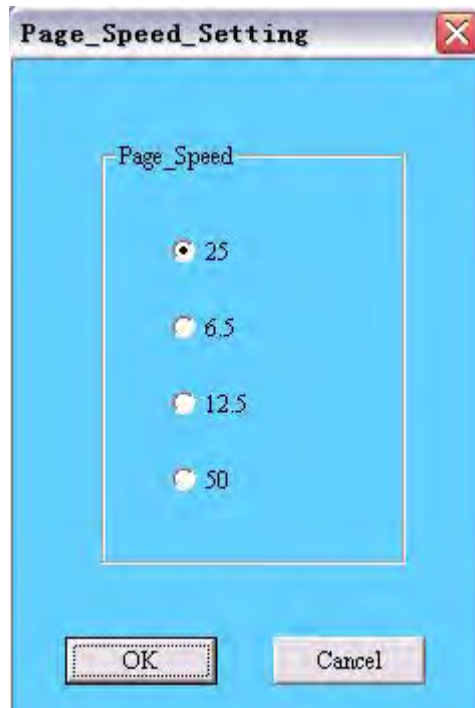

The user can select one of Page Speeds in the above figure and click **OK** key. Then ECG wave in Print Interface will be displayed with the selected speed.

### 4.12.3 Gain Setting


Click **Gain** key in the Main tool bar of Print Interface in Figure 4-9 to pop up the Gain Setting Interface.

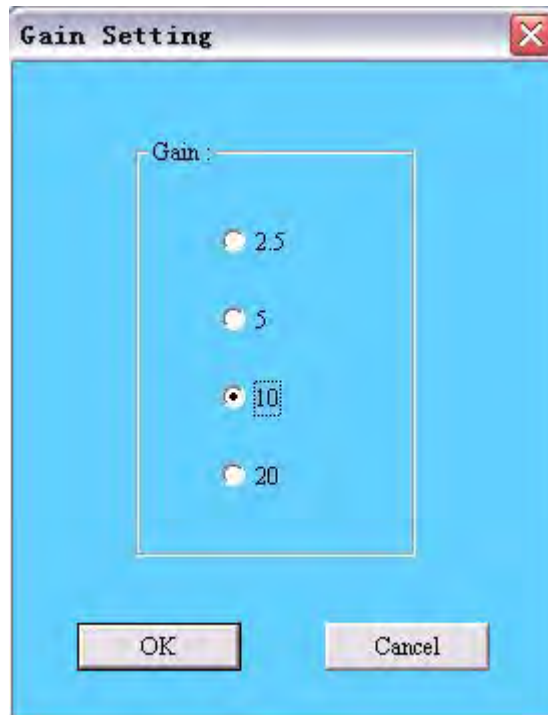

The user can select one of Gains in the above figure and click **OK** key. Then ECG wave in Print Interface will be displayed with the selected gain.

## 4.12.4 Rhythm Lead Select

Click **Rhythm Lead** key in the Main tool bar of Print Interface in Figure 4-9 to pop up the Rhythm Lead Select Interface.

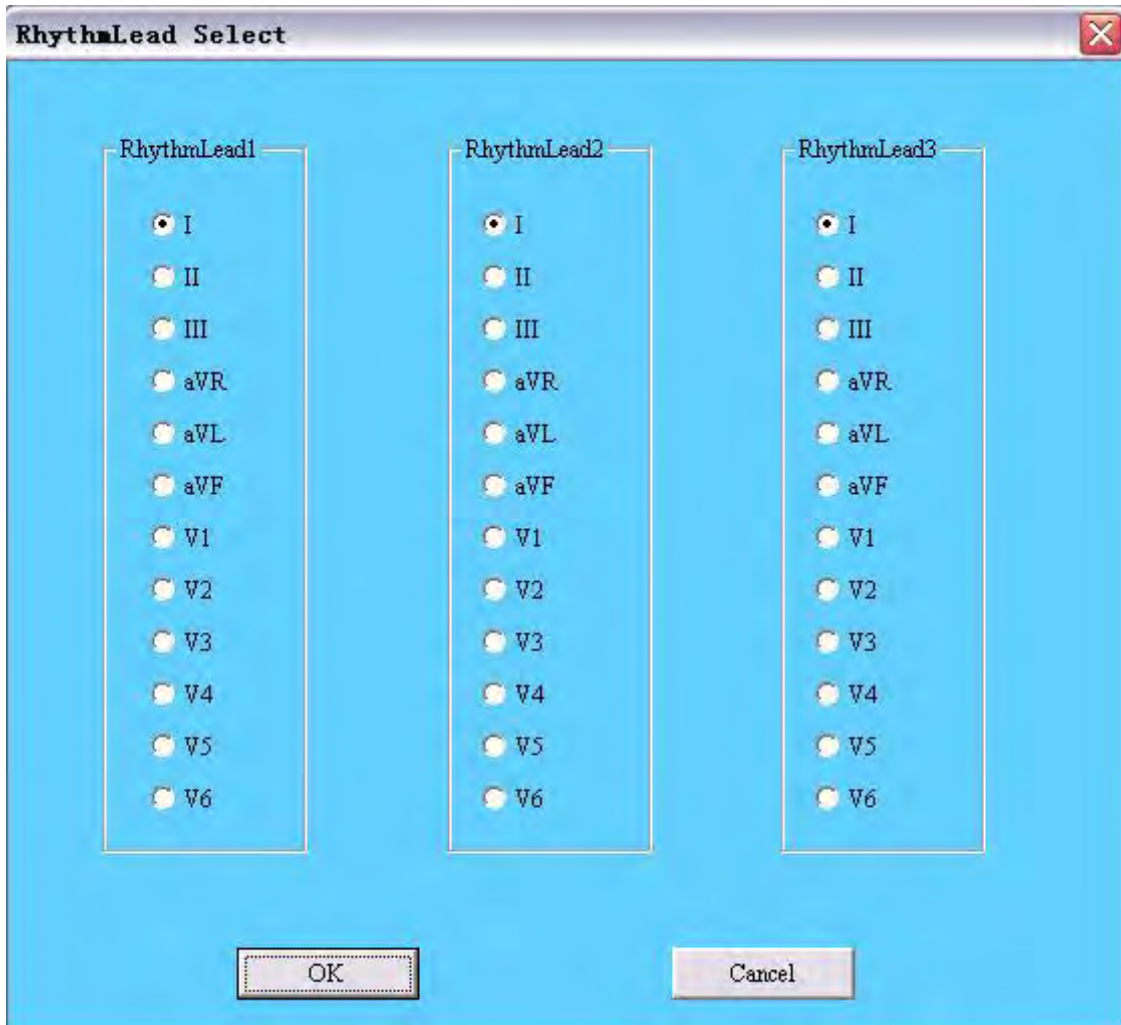


The rhythm lead can be one of 12 standard leads: I, II, III, aVR, aVL, aVF, V1, V2, V3, V4, V5, or V6.

When Display Format is set as 3\*4+1 or 6\*2+1, the rhythm lead selected in Rhythm Lead 1 will be displayed and recorded; when Display Format is set as 3\*4+3, 3 rhythm leads selected in Rhythm Lead 1 and Rhythm Lead 2 and Rhythm Lead 3 will be displayed and recorded.

## 4.12.5 Sequence Select

Click **Sequence** key in the Main tool bar of Print Interface in Figure 4-9 to pop up the Sequence Select Interface.

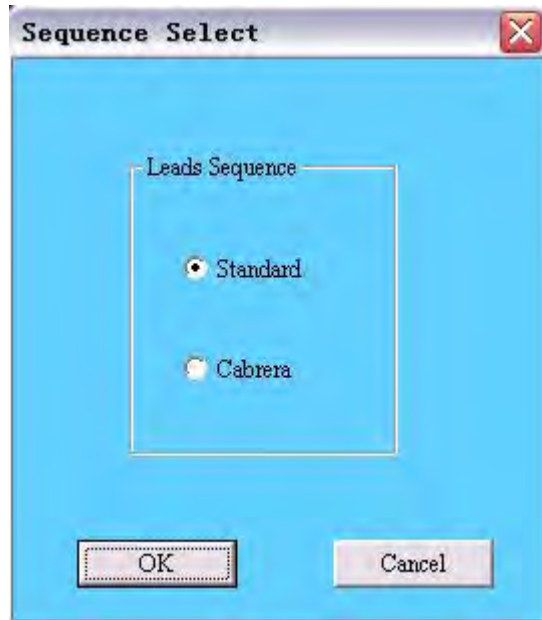

The user can select one of Lead Sequences in the above figure and click **OK** key. Then ECG wave in Print Interface will be displayed with the selected sequence.


### Lead Sequence: Standard/Cabrera

Lead Sequence	Lead group 1	Lead group 2	Lead group 3	Lead group 4
<b>Standard</b>	I, II, III	aVR, aVL, aVF	V1, V2, V3	V4, V5, V6
<b>Cabrera</b>	aVL, I, -aVR	II, aVF, III	V1, V2, V3	V4, V5, V6



## 4.12.6 Print Content Select


Click **Content** key in the Main tool bar of Print Interface in Figure 4-9 to pop up the Print Content Select Interface.

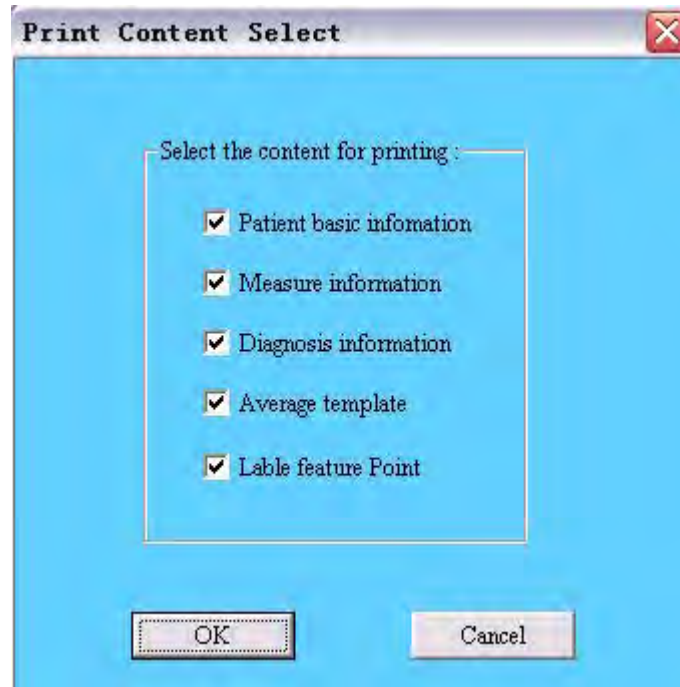

The following figure shows the Print Interface when **Patient basic information** item and **Diagnosis information** item are not selected. And the content of the two items will not be printed on the paper.

**Note:** Average Template is not selected in default situation. When selected, average template will be previewed, printed or created into PDF file as an independent page. If not selected, when previewing, printing or creating PDF file, there will be only one page and no the second page of average template.

## Smart ECG-Viewer Data Management Software User Manual


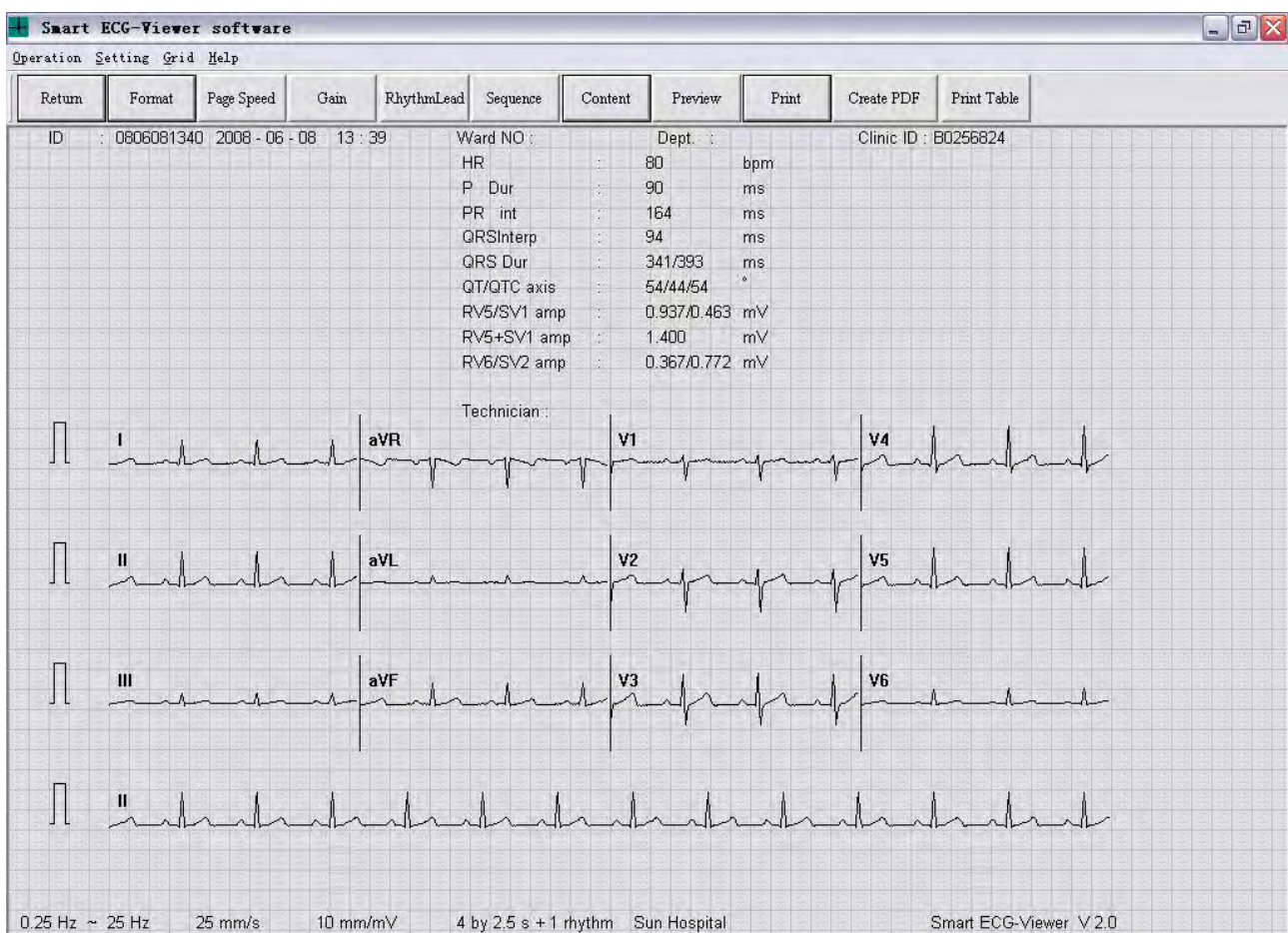


### 4.12.7 Preview


Click **Preview** key in the Main tool bar of Print Interface in Figure 4-9 to pop up the Preview Interface.



# Smart ECG-Viewer Data Management Software User Manual

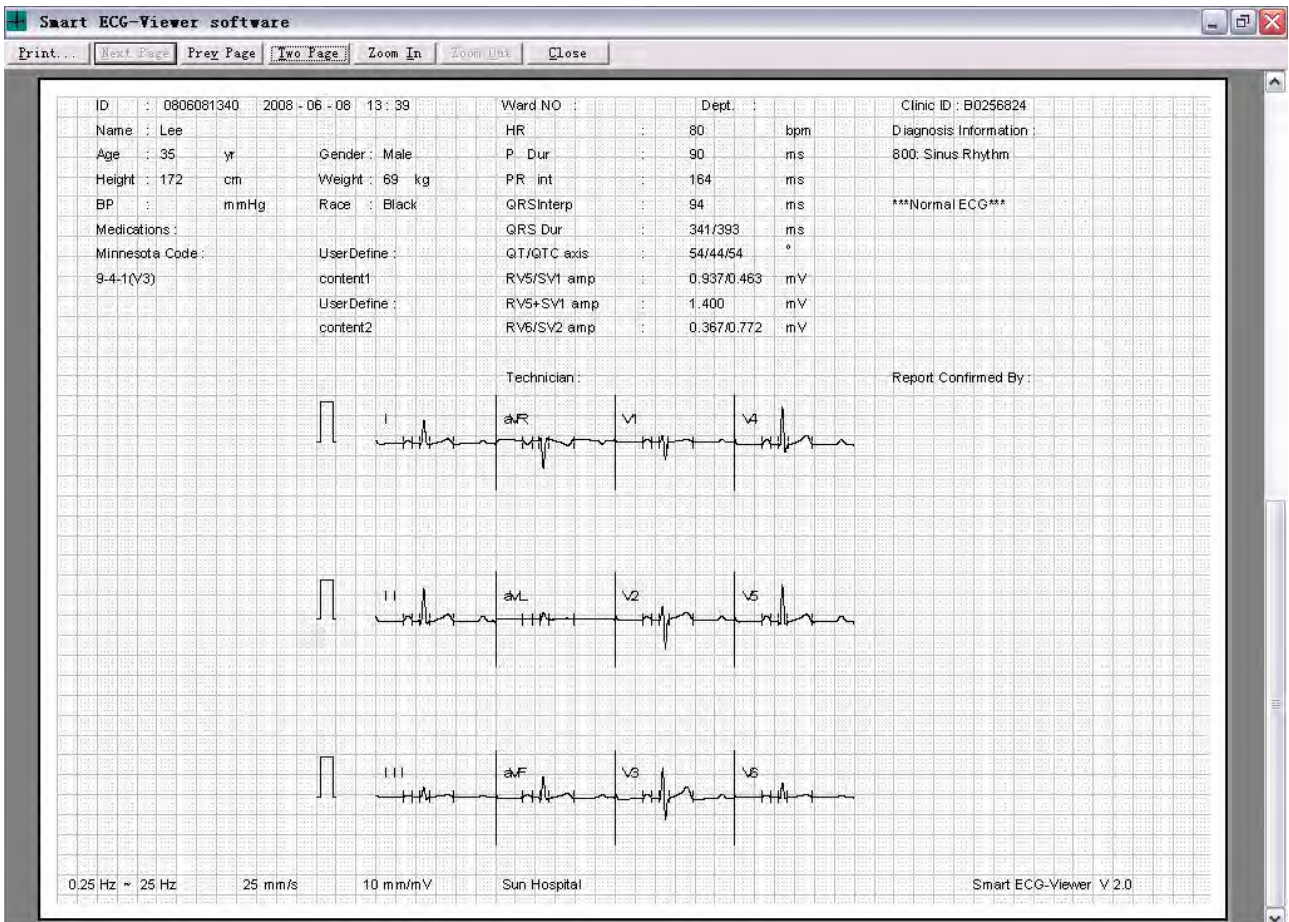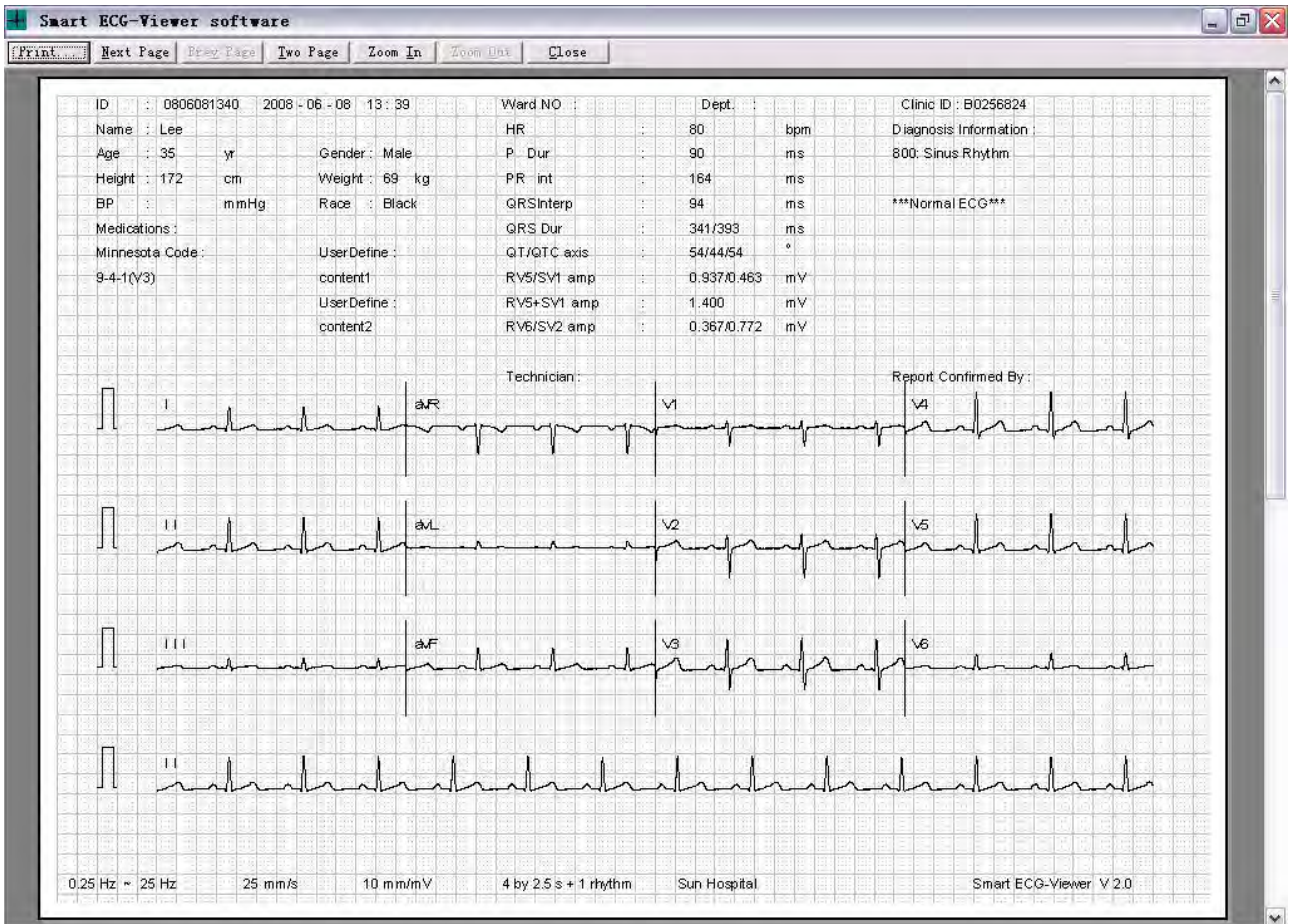

In the above Preview Interface, click **Next Page/Prev Page** key to switch to the next/pre preview page; click **Two Page** key to preview the two preview pages in one screen simultaneously; Click **Zoom In** key to magnify the preview page; Click **Zoom Out** key to minify the preview page; Click **Print** key to pop up the Printer Setting Interface shown below; Click **Close** key to close Preview Interface and return to the Print Interface.

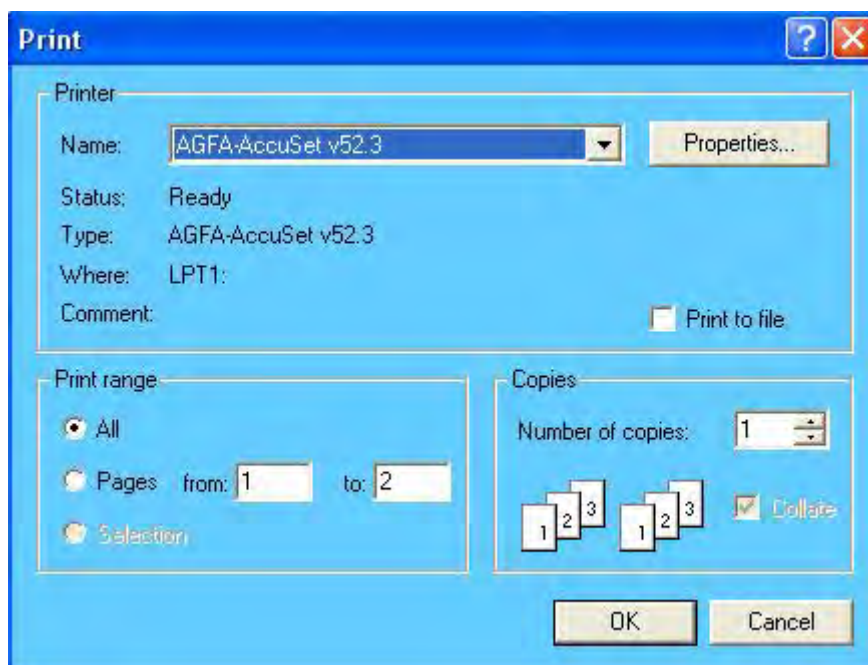

Set the printer and click **Ok** key to print.

**Note:** Do not select the PDF creator as a printer, which is a dummy printer.


### 4.12.8 Print

Click **Print** key in the Main tool bar of Print Interface in Figure 4-9 to pop up the Printer Setting Interface shown above. Set the printer and click **Ok** key to print.

**Note:** Do not select the PDF creator as printer, which is a dummy printer.


### 4.12.9 Create PDF

Click **Create PDF** key in the Main tool bar of Print Interface in Figure 4-9 to pop up the Create PDF Interface shown below.

After set the File Name and Save Path, click **OK** key to begin to create PDF file. And then the icon  will be displayed in the bottom right corner of the desk till the PDF file has been created successfully. After created successfully, the hint information will be displayed.

**Note:** Creating a PDF file will cost some dozen seconds, and the user should be patient to wait.




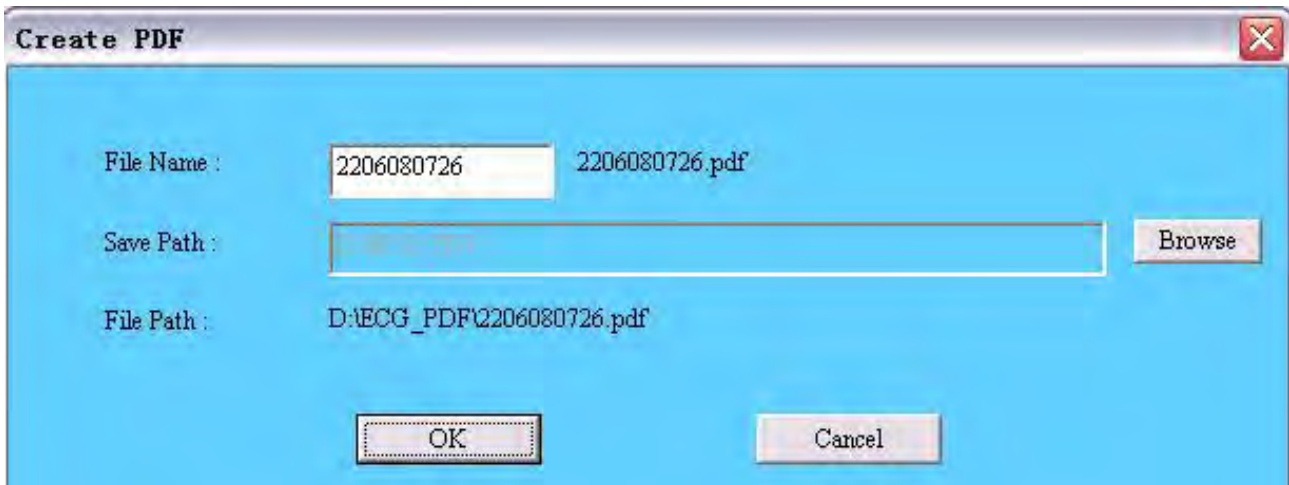


When the PDF file has been created successfully, the user can view it in the storage position.

#### 4.12.10 Print Table

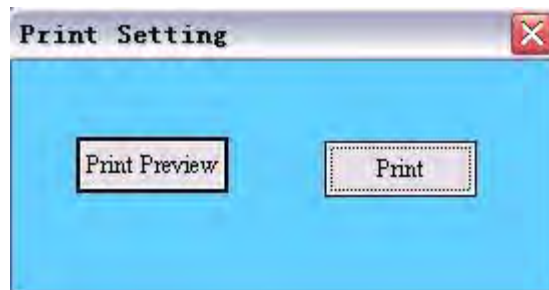

Click **Print Table** key in the Main tool bar of Print Interface in Figure 4-9 to pop up the following dialog box.

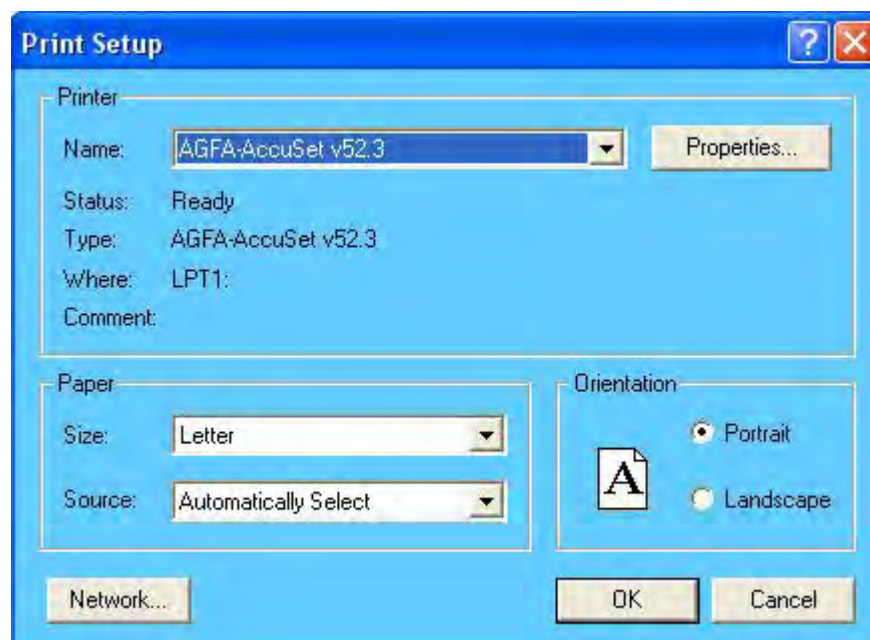


Click **Print Preview** key to pop up the following figure.



Set the printer and paper size and click **Ok** key to display the following interface. The user can drag

the scroll bar on the right to preview the whole table. Click the icon  on the tool bar to print this table; click the icon  on the tool bar to return to the Print Interface.

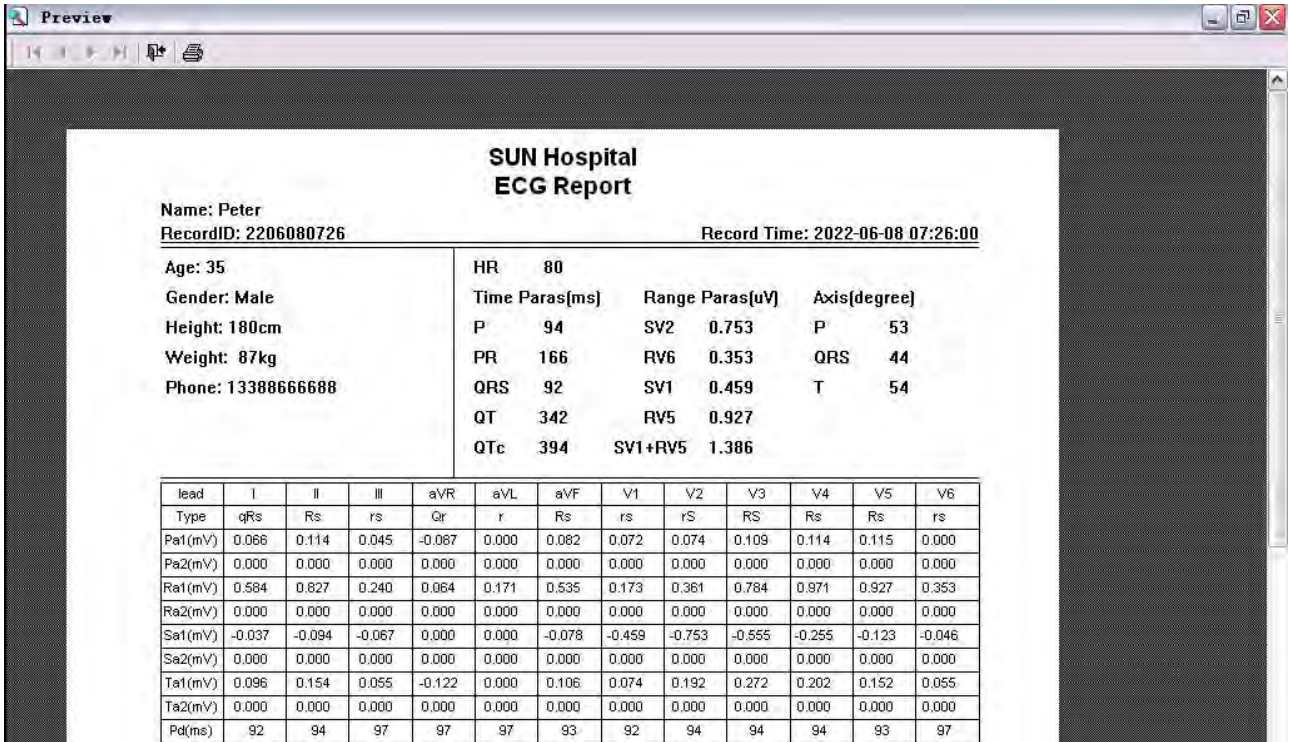

**SUN Hospital  
ECG Report**

Name: Peter  
RecordID: 2206080726  
Record Time: 2022-06-08 07:26:00

Age: 35	HR: 80
Gender: Male	Time Paras(ms): 94
Height: 180cm	Range Paras(uV): SV2 0.753
Weight: 87kg	Axis(degree): P 53
Phone: 13388666688	QRS: 44
	QT: 342
	QTc: 394
	SV1: 0.459
	RV5: 0.927
	SV1+RV5: 1.386

lead	I	II	III	aVR	aVL	aVF	V1	V2	V3	V4	V5	V6
Type	qRs	Rs	rs	Qr	r	Rs	rs	rS	RS	Rs	Rs	rs
Pa1(mV)	0.066	0.114	0.045	-0.087	0.000	0.062	0.072	0.074	0.109	0.114	0.115	0.000
Pa2(mV)	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000
Ra1(mV)	0.584	0.827	0.240	0.064	0.171	0.535	0.173	0.361	0.784	0.971	0.927	0.353
Ra2(mV)	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000
Sa1(mV)	-0.037	-0.094	-0.067	0.000	0.000	-0.078	-0.459	-0.753	-0.555	-0.255	-0.123	-0.046
Sa2(mV)	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000
Ta1(mV)	0.096	0.154	0.055	-0.122	0.000	0.106	0.074	0.192	0.272	0.202	0.152	0.055
Ta2(mV)	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000
Pd(ms)	92	94	97	97	97	93	92	94	94	94	93	97

The detailed content of the table is shown below.

**Note:** If the user does not enter the hospital name and address, the two items will not be displayed in the ECG report.

## SUN Hospital ECG Report

Name: Peter

RecordID: 2206080726

Record Time: 2022-06-08 07:26:00

Age: 35

Gender: Male

Height: 180cm

Weight: 87kg

Phone: 13388666688

HR 80

Time Paras(ms)      Range Paras(uV)      Axis(degree)

P 94      SV2 0.753      P 53

PR 166      RV6 0.353      QRS 44

QRS 92      SV1 0.459      T 54

QT 342      RV5 0.927

QTc 394      SV1+RV5 1.386


lead	I	II	III	aVR	aVL	aVF	V1	V2	V3	V4	V5	V6
Type	qRs	Rs	rs	Gr	r	Rs	rs	rS	RS	Rs	Rs	rs
Pa1(mV)	0.066	0.114	0.045	-0.087	0.000	0.082	0.072	0.074	0.109	0.114	0.115	0.000
Pa2(mV)	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000
Ra1(mV)	0.584	0.827	0.240	0.064	0.171	0.535	0.173	0.361	0.784	0.971	0.927	0.353
Ra2(mV)	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000
Sa1(mV)	-0.037	-0.094	-0.067	0.000	0.000	-0.078	-0.459	-0.753	-0.555	-0.255	-0.123	-0.046
Sa2(mV)	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000
Ta1(mV)	0.096	0.154	0.055	-0.122	0.000	0.106	0.074	0.192	0.272	0.202	0.152	0.055
Ta2(mV)	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000	0.000
Pd(ms)	92	94	97	97	97	93	92	94	94	94	94	93
Qd(ms)	22	0	0	44	0	0	0	0	0	0	0	0
Rd1(ms)	50	65	60	36	89	64	47	50	54	60	64	67
Rd2(ms)	0	0	0	0	0	0	0	0	0	0	0	0
Td(ms)	114	119	118	114	117	119	116	115	116	116	116	119
PR(ms)	165	166	167	167	167	166	166	165	167	166	167	166
QT(ms)	342	343	342	341	341	344	341	343	342	342	343	344
QRS(ms)	438	439	439	439	439	438	439	438	439	438	439	438
VAT(ms)	12	40	38	40	0	41	37	38	39	40	40	40
STj(mV)	-0.034	-0.054	-0.016	0.040	-0.011	-0.035	-0.003	-0.016	-0.011	-0.043	-0.056	-0.032
ST1(mV)	-0.024	-0.032	0.000	0.027	-0.011	-0.016	0.018	0.047	0.024	-0.011	-0.027	-0.019
ST2(mV)	0.000	-0.003	0.008	-0.005	-0.011	0.002	0.031	0.063	0.060	0.026	0.003	-0.006
ST3(mV)	0.022	0.032	0.013	-0.029	-0.011	0.024	0.039	0.090	0.111	0.063	0.038	0.008
ST20(mV)	-0.032	-0.040	-0.005	0.032	-0.016	-0.022	0.015	0.034	0.013	-0.025	-0.037	-0.024
ST40(mV)	-0.018	-0.030	0.000	0.019	-0.013	-0.014	0.020	0.050	0.032	-0.006	-0.021	-0.016
ST60(mV)	-0.005	-0.013	0.003	0.000	-0.008	-0.006	0.026	0.061	0.050	0.016	-0.002	-0.011
ST80(mV)	0.014	0.013	0.013	-0.019	-0.005	0.013	0.034	0.074	0.082	0.042	0.016	0.000

Report Date:2007-09-29

## 4.13 User Management


**Note:** When installing the system, the default user *su* is an administrator. The administrator can add, manage and delete ordinary users, but he can not delete himself. When delete ordinary users, the administrator should provide his own password. Ordinary users have the right to modify their passwords.

Click **【Setting】** à **【User Setting】** in the main menu bar to pop up the User Setting dialog box, as the following figure shows, and the user can add or delete system user or modify password in it.

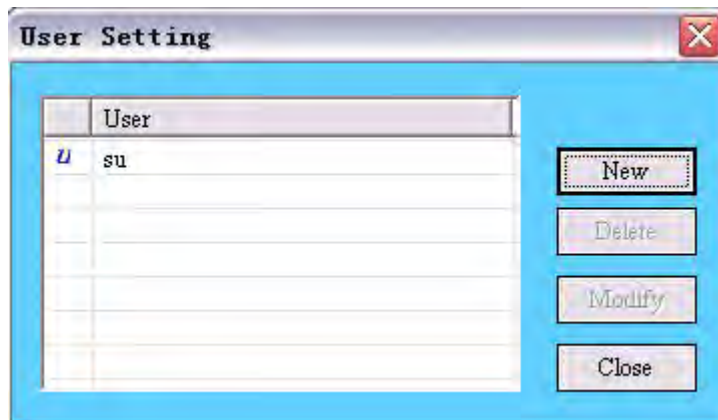

### 4.13.1 New User

Click **New** button in the User Setting dialog box to pop up the New User dialog box. After input the user's information and click **OK**, a new user can be added in the system.

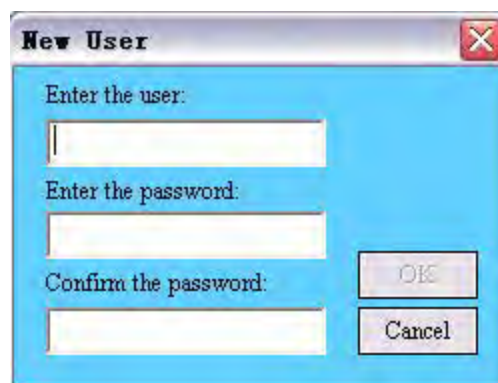

**Note:** The user name should be exclusive. If the inputted name is repetitious with the name that has been existed in the system, the prompt window will pop up to remind the user to change another name.



### 4.13.2 Delete User

Select one user in the User Setting dialog box and click **Delete** to pop up the Delete User dialog box.


After input the right password for this user and click **OK**, this user can be deleted.



### 4.13.3 Modify Password

Select one user in the User Setting dialog box and click **Modify** to pop up the Modify Password dialog box. After enter the old password, the new password and confirm the new password and click

**OK**, the password will be modified successfully.

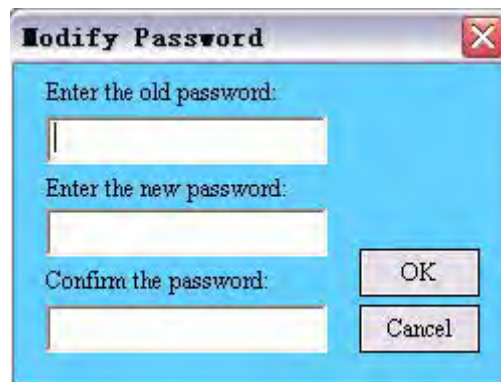


### 4.14 Time Format Setting

Click **【Setting】** à **【Time Format】** in the main menu bar to select one of time formats: yyyy-mm-dd, mm-dd-yyyy or dd-mm-yyyy. Then the time display format in the software will change to the selected one.

## 4.15 User Defined Setting

Click **【Setting】** à **【User Defined Setting...】** in the main menu bar to pop up the User Defined dialog box, as the following figure shows.

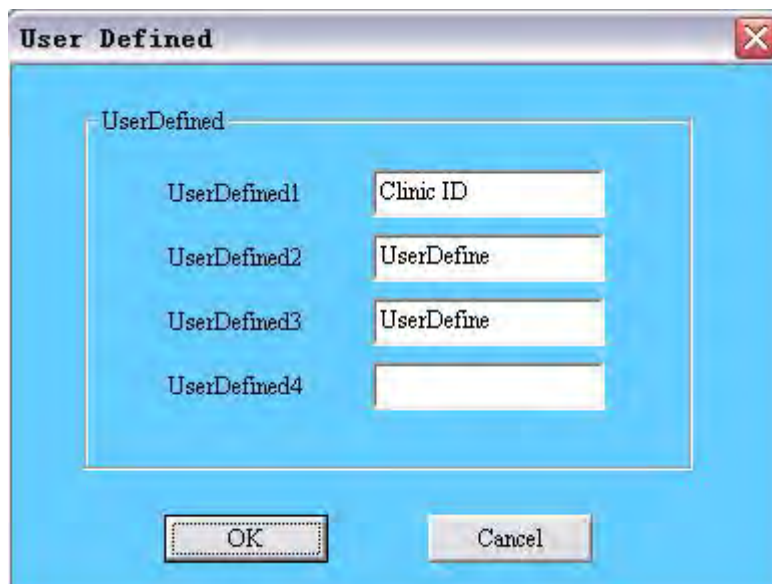

When the user fills in user-defined items in UserDefined1/2/3/4 in the above figure and click OK to confirm, the user-defined items will be displayed in the bottom right corner of Patient Infos Window.

For example: In the above User Defined dialog box, enter “Clinic ID” in UserDefined1, enter “UserDefine” in UserDefined2, enter “UserDefine” in UserDefined3 and leave UserDefined4 blank. Click OK to confirm the previous operations. Then select a patient record in Patient Records Window (See Figure 4-1) and view its Patient Infos Window. The user will find that “Clinic ID”, “UserDefine” and “UserDefine” filled in the above figure are displayed in the bottom right corner of Patient Infos Window, as the following figure shows.

**Patient Infos**

**Patient Basic Info**

No: 07112117022300    Gender:  Male  Female

Name: Lee

Age: 35 Years    Race: Black

Phone:    Height(cm): 172

Dept.:    Weight(kg): 69

Ward:    BP(mmHg): /

Drug:    Time: 08/06/08 13:39

Address:    Technician:

Disease History:

**Diagnose**

Doctor: Jakie   

Phone:

Dept.:

Hospital: Sun Hospital

**User Defined**

Clinic ID: B0256824

UserDefine: content1

UserDefine: content2

When the user-defined items are displayed in the bottom right corner of Patient Infos Window, the user can fill in the corresponding information of the patient into these items. And when previewing the print content, the user-defined items and corresponding information filled by the user will be displayed in the Print Interface.

For example: In the above Patient Infos Window, enter “B0256824” in Clinic ID, enter “Content1” in UserDefine and enter “Content2” in the second UserDefine. Click Modify to confirm the previous operations. When access the Print Interface, the user will find that “Clinic ID: B0256824”, “UserDefine: Content1” and “UserDefine: Content2” are displayed in the Print Interface, as the following figure shows.

# Smart ECG-Viewer Data Management Software User Manual

User Defined 2

User Defined 1

The screenshot displays the 'Smart ECG-Viewer software' interface. At the top, there are menu options: 'Operation', 'Setting', 'Grid', and 'Help'. Below this is a toolbar with buttons for 'Return', 'Format', 'Page Speed', 'Gain', 'RhythmLead', 'Sequence', 'Content', 'Preview', 'Print', 'Create PDF', and 'Print Table'. The main area is divided into two sections: a data table and an ECG display.

ID	: 0806081340	2008 - 06 - 08 13 : 39	Ward NO :	Dept. :	Clinic ID : B0256824
Name	: Sanming		HR	: 80 bpm	Diagnosis Information :
Age	: 35 yr	Gender: Male	P Dur	: 90 ms	800: Sinus Rhythm
Height	: 180 cm	Weight: 68 kg	PR int	: 164 ms	
BP	: mmHg	Race :	QRSInterp	: 94 ms	***Normal ECG***
Medications :			QRS Dur	: 341/393 ms	
Minnesota Code :	9-4-1(V3)	UserDefine :	QT/QTc axis	: 54/44/54 °	
		Content1	RV5/SV1 amp	: 0.937/0.463 mV	
		UserDefine :	RV5+SV1 amp	: 1.400 mV	
		Content2	RV6/SV2 amp	: 0.367/0.772 mV	

Below the table, there are fields for 'Technician :', 'Report Confirmed By :', and a grid of ECG traces for leads I, aVR, V1, V4, II, aVL, V2, V5, III, aVF, V3, V6, and a rhythm strip. The bottom status bar shows: '0.25 Hz ~ 25 Hz', '25 mm/s', '10 mm/mV', '4 by 2.5 s + 1 rhythm', 'Sun Hospital', and 'Smart ECG-Viewer V 2.0'.

User Defined 3

User Defined 4

## 4.16 Grid Color Setting

The user can set the color of the grid on the background of interface when viewing ECG curve, browsing rhythm data, browsing multi-cases rhythm data and previewing ECG printing.


Click **Grid** à **Modify Color** in the main menu bar to pop up the following figure. The user can select one of the Basic colors in the following figure and click OK to confirm. And then the color of grid will change to the one selected.



Click **Define Custom Colors >>** button to display the following figure. And then the user can define custom color in the following figure.

Click on the Part 3 and move the **Black Triangle Marker** of part 4 to get the color you want, which is displayed in **Color|Solid**. Then click **Add to Custom Colors** button to add this color to Part 1 and click OK to confirm. The grid color will change to the color defined.



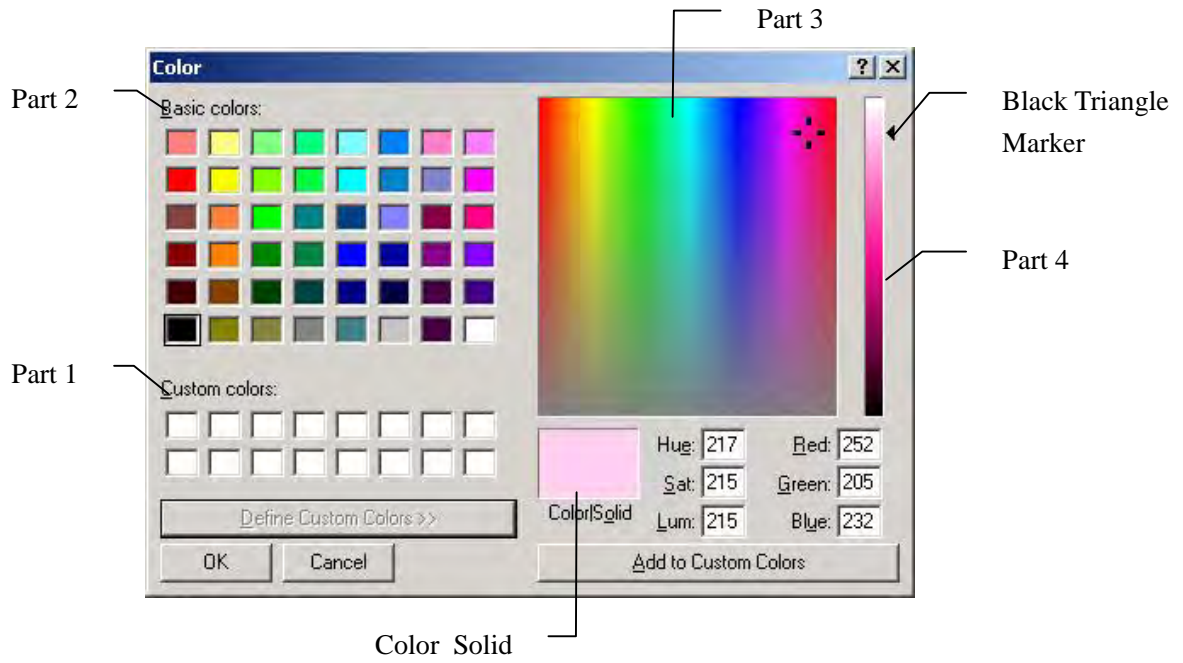

If the user wants to recover the default color, he can click **【Grid】** à **【Default Color】** in the main menu bar to get it.



## 5 Maintenance and Service

The user should be familiar with all the operation steps and use methods of this system device. Read this manual in detail, and don't use this device before understand the whole operation process.



### 5.1 Maintenance

Smart ECG-Viewer Data Management Software just needs simple maintenance during the using process. But only the right operation can guarantee the system's long-time、 steady working. So the user must conform to the Notice and Maintenance presented by manufacturer strictly.

 **WARNING** : Power off before cleaning.

#### Maintenance for the Main Unit and Monitor

- 1) Keep the main unit and the monitor clean. The main unit case can be cleaned with a soft cloth dampened with water. If necessary, use a mild detergent but carefully remove any residue. Use water very sparingly to prevent liquid from entering the equipment.
- 2) The main unit and the monitor should be placed in the dry and well-ventilated place. Avoid being placed in the dusty and humid environment. The air path for cooling the system should be kept well ventilated.

 **CAUTION** : Avoid pouring liquids on the equipment while cleaning, and do not immerse any parts of the equipment into any liquids.

## 5.2 Service Information

For service, maintenance and any questions regarding this system, please contact the manufacturer or your local distributor.

**Manufacturer: EDAN INSTRUMENTS, INC.**

**Addr. :** 2/F, block A/B, Unit 8, Xing Hua Building (6th Industrial Road, Shekou), Nanhai Road,  
Nanshan District, 518067 Shenzhen PEOPLE'S REPUBLIC OF CHINA

**Zip Code:** 518067

**Tel:** +86-755-26897203, 26897216

**Fax:** +86-755-26897201

**E-Mail:** [service@edan.com.cn](mailto:service@edan.com.cn)



**EDAN INSTRUMENTS, INC.**

3/F - B, Nanshan Medical Equipments Park, Nanhai Rd 1019#, shekou, Nanshan  
Shenzhen, 518067 P.R. China

TEL:86-755-26882220 FAX: +86-755-26882223