Electrodes placement manual

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GB

Please refer to the photographs to check where the electrodes should be placed for each of the various treatments. It is essential that the electrodes adhere perfectly to the skin: it is therefore advisable to clean the skin in the area to be treated. Warning: particularly hairy areas will be insulated; in order to achieve the correct stimulation, it is advisable, in some cases, to depilate this area. If the electrodes are worn, they must be replaced.

Contra-indications

Do not use this appliance if any of the following conditions apply: persons with a pace-maker, persons suffering from heart disease, epileptics, pregnant women, persons with active phlebitis, persons with a fever, or with thrombo-phlebitis, persons suffering from anxiety, or from any serious diseases.

Should there be any muscular trauma or stress, or any other health problem, only use the product after consulting your doctor and use it under medical supervision.

Side effects

No significant side effects are known. In some cases, in particularly sensitive persons, the skin where the electrodes are placed can become reddened: this redness normally disappears a few minutes after the treatment. If the redness persists, contact a doctor. In rare cases, stimulation in the evening can lead to some persons subsequently taking longer to fall asleep. In this case, suspend treatment and consult a doctor.

Warnings

- It is prohibited to place the electrodes so that the current flows across the cardiac area (e.g. a black electrode on the chest and a red electrode on the shoulder blade); however, it is permissible to place the electrodes along the muscle groups in the cardiac area, as in pectoral strengthening.

- It is prohibited to use the apparatus with the electrodes placed on or near lesions or cuts on the skin.
- It is prohibited to place the electrodes on the carotid sinuses (carotid), or on the genitals.
- It is prohibited to place the electrodes near the eyes or to direct the current at the eyeball (by placing one electrode diametrically opposite the other in the eye area); keep at least 3 cm. away from the eyeball.
- Follow the instructions in this manual and on the packaging of the electrodes, when using the electrodes. Use single-patient electrodes, supplied exclusively by the producer or retailer, and strictly avoid the exchange of electrodes between different users.
- Only use electrodes supplied by the manufacturers; the appliance is tested and guaranteed for use with the electrodes that are supplied with it.
- Electrodes of inadequate size can provoke reactions on the skin or even burns. Only use the round electrodes supplied with the appliance for treatment to the face.
- Do not use the electrodes if they are damaged, even if they adhere well to the skin.
- Should an electrode no longer adhere to the skin, do not use it. Repeated use of the same electrodes can jeopardize the safety of the stimulation, and can in fact cause a reddening of the skin that lasts for several hours after the stimulation has finished.
- Warning: during treatment with the round electrodes and those measuring 35x45mm. it is possible to exceed a current density of 2mA/cm² for each electrode. In these cases, pay particular attention to any redness of the skin.
- Should any skin redness persist for several hours after the end of treatment, stop the applications and consult your doctor.

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WARNING: Wet the bands before use (submerge them in warm water).

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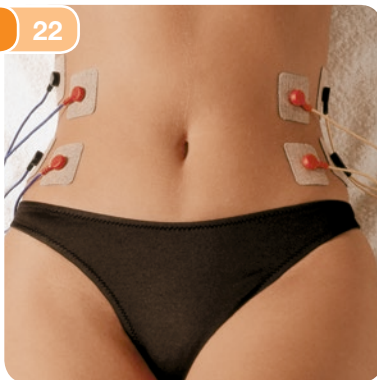
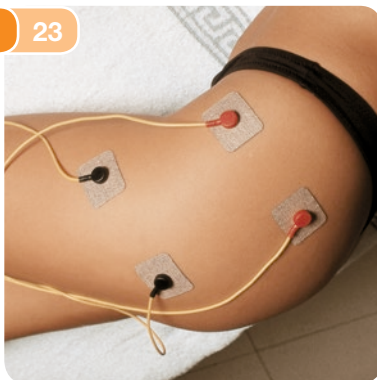
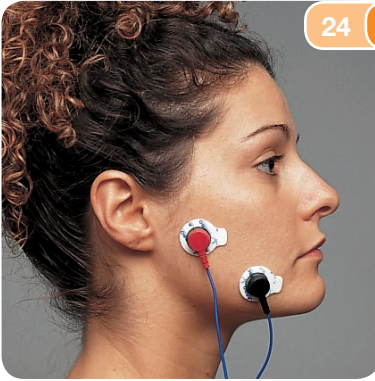
WARNING: It is prohibited to place the electrodes near the eyes or to direct the current at the eyeball (by placing one electrode diametrically opposite the other in the eye area); keep at least 3 cm away from the eyeball.

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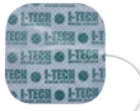
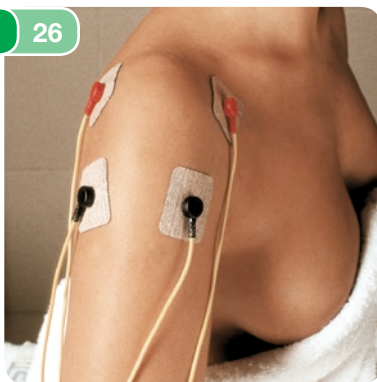
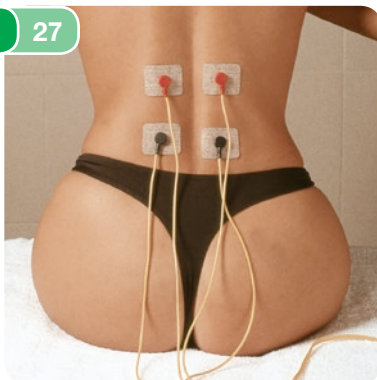
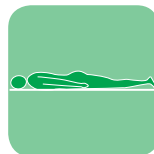
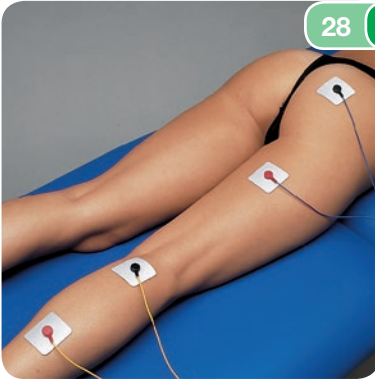
48 x 48 mm

41 x 41 mm
48 x 48 mm

WARNING:
Do not use the
splitter cables to
avoid short circuit

29

Gomito (epicondilite)

(per programma medicale)

- (GB) Area: elbow (epicondylitis)
(for medical programme)
- (F) Zone: coude (épicondylite)
(pour programme médical)
- (D) Bereich: Ellenbogen (Tennisarm)
(medizinisches Programm)
- (E) Zona: codo (epicondilitis)
(para programa médico)
- (NL) Zone: elleboog (epicondilitis)
(geneeskundig programma)

48 x 48 mm

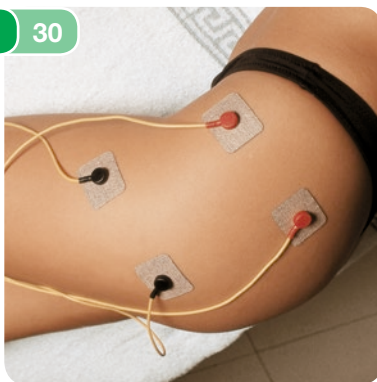
41 x 41 mm
48 x 48 mm

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30

(per programma medicale)

- Ⓒ Area: hip (hip pain)
(for medical programme)
- Ⓕ Zone: hanche (douleurs à la hanche)
(pour programme médical)
- Ⓖ Bereich: Becken (Beckenschmerzen)
(medizinisches Programm)
- Ⓔ Zona: cadera (dolores de cadera),
(para programa médico)
- Ⓗ Zone: heup (heuppijn)
(geneeskundig programma)

41 x 41 mm
48 x 48 mm

48 x 48 mm

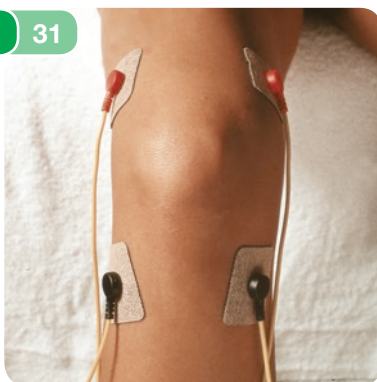
Ginocchio (dolori al ginocchio)

31

(per programma medicale)

- Ⓒ Area: knee (knee pain)
(for medical programme)
- Ⓕ Zone: genou (douleurs au genou)
(pour programme médical)
- Ⓖ Bereich: Knie (Knieschmerzen)
(medizinisches Programm)
- Ⓔ Zona: rodilla (dolores de rodilla)
(para programa médico)
- Ⓗ Zone: knie (kniepijn)
(geneeskundig programma)

41 x 41 mm
48 x 48 mm

48 x 48 mm

32

Caviglia (distorsione/contusione)

(per programma medicale)

(GB) Area: ankle (sprains/bruising)
(for medical programme)

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Polso (tunnel carpale)

(per programma medicale)

(GB) Area: wrist (carpal tunnel)
(for medical programme)

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IACER Srl
I marchi I-TECH / MIO CARE nella loro veste grafica sono proprietà di IACER S.r.l.