

Simply pull over the AquaProtect –
without any tape or rubber strap.



In case of a rough surface
pull over a stocking or
similar in order to avoid
splitting.

Cover open wounds.

AquaProtect is extremely
elastic - just pull it as high
as possible.



Pull back slightly and
make seal lie flat to the
skin of the muscle area.

If you feel the seal is
constrictive and/or it rolls
up at the top, reduce

tension by stretching it
forcibly several times.

Ready for taking a shower.

For bathing and swimming
immerse carefully the
major part of the

AquaProtect. Remove

excess air inside by pulling



a corner and re-establish seal. Ready for bathing or swimming.

AquaProtect®

Description	Model	Length ±	Circumference of the Extremity
Half Arm, large	AP 21	54 cm	≥ 22 cm
Half Arm, small	AP 17	43 cm	≥ 17 cm
Full Arm, large	AP 31	80 cm	≥ 26 cm
Full Arm, small	AP 26	66 cm	≥ 19 cm
Half Leg, large	LP 25	64 cm	≥ 32 cm
Half Leg, small	LP 20	51 cm	≥ 24 cm
Full Leg, large	LP 44	112 cm	≥ 47 cm
Full Leg, small	LP 33	84 cm	≥ 31 cm



Wherever you'll be:
in bath, pool or sea –
AquaProtect(s) your bandage!

AquaProtect®

 **PRAMETA**

Genker Strasse 16 • D-53842 Troisdorf
Tel. +49(0)2241.23996-0 • Fax +49(0)2241.23996-33
info@praemeta.de • www.praemeta.de 

03/07 • Subject to technical modifications. Misprintings and other errors excepted from guarantee.

AquaProtect®

Models for half
and full arm.



AquaProtect makes living with a bandage much more comfortable.

A refreshing shower or even a relaxing full-bath – now it's really easy.

Dish washing or wiping the floor does not mean a problem any longer: Pull AquaProtect over the bandage and get a 100% water proof protection.

Models for half
and full leg



Wherever you'll be:
in bath, pool or sea –
AquaProtect(s)
your bandage!

AquaProtect is a protection cover made of skinfriendly polyurethane.

There are eight different models: two sizes each (large and small) for half arm, full arm, half leg and full leg.

The leg models have slip-safe soles providing hold and security on slippery floors.


AquaProtect does not contain any Latex and is very tough. This means: no danger of allergic problems, no danger of tear.

AquaProtect can be applied many times.

Get AquaProtect in your pharmacy or at your Medical store.

**Aqua
Protect**

All models in large
and small sizes

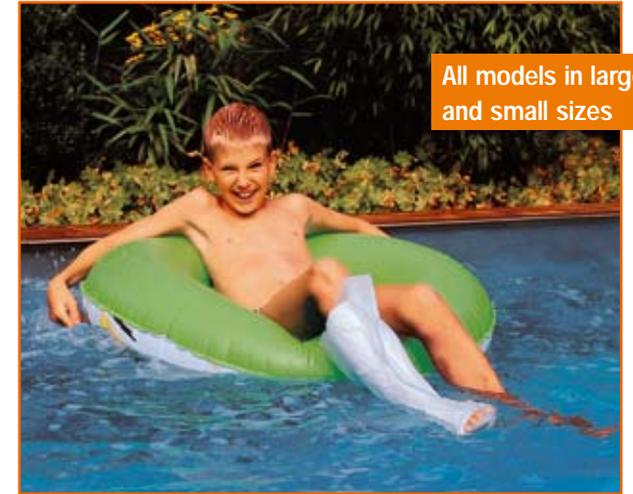

AquaProtect protects efficiently bandages on arms and legs - not only for taking a shower, but also while bathing and swimming in the pool or in the sea.

It does not matter whether you are swimming or diving

– AquaProtect provides a watertight seal.

Suitable for
sports as well

