

HISTOFREEZER® CRYOSURGICAL SYSTEM

- **30591 HISTOFREEZER® - kit with applicator 2 mm**
Two 80 ml aerosol bottles and 60 applicators 2 mm.
- **30592 HISTOFREEZER® - kit with applicator 5 mm**
Two 80 ml aerosol bottles and 52 applicators 5 mm.
- **30589 HISTOFREEZER® - large kit mixed applicators**
Two 80 ml aerosol bottles, 24 applicators 2 mm, 36 applicators 5 mm.
- **30585 HISTOFREEZER® - mini kit mixed applicators**
One 80 ml aerosol bottle, 16 applicators 2 mm, 16 applicators 5 mm.

The Histofreezer® portable cryosurgical system is a low cost, highly effective cryosurgical agent that provides safe and effective treatment for warts and other common benign skin lesions.

This therapeutic treatment makes use of local freezing for the controlled destruction or removal of living tissue.

Suitable for adults and children

Effective use on 9 benign lesion types

	% of resolution*
- Verruca Vulgaris (Common Warts)	94%
- Verruca Plantaris (Plantar Warts)	86%
- Molluscum Contagiosum	100%
- Lentigo (Age Spots)	100% - facial 99% - non facial
- Condyloma(Genital Warts)	89%
- Seborrhic Keratosis	95%
- Acrochordon (Skin Tags)	100%
- Verruca Plana (Flat Warts)	93%
- Actinic keratosis (Facial and Non-Facial)	91% - facial 87% - non facial

*Summary of resolution rates by type of lesion from a number of clinical trials obtained with histofreezer + post-treatment of 27% salicylic acid / 17% lactic acid
Supplied with box and manual in: GB, FR, IT, DE, ES, NL, PL.
From web you can download: GR, PT, TH, Arab.

Saturated 2 mm applicators will stay cold for 90 seconds.



30591



30591

Saturated 5 mm applicators will stay cold for 120 seconds.



30585



30592



30589



2 mm
5 mm

2 mm disposable applicator for small lesions
5 mm disposable applicator for large lesions

CHARGE
custom applicator press valve until droplets start to form.


INVERT
canister for 15 seconds for optimal freezing temperature.

APPLY
a frozen applicator to lesion for 15-40 seconds.

BIOLOGICAL MICROSCOPES - MAGNIFICATION 40X-1600X

- **31002 LED BIOLOGICAL MICROSCOPE - 40X to 1600X**
High level microscope with high resolution and transmission
Main features:
- infinite optical system
- wide field eyepiece WF 10X/20
- infinite semi-plan achromatic objectives 4X, 10X, 40X, 100X
- Abbe condenser
- backward quadruple nosepiece
- double layers mechanical stage (size: 150x139 mm, move range 75x52 mm)
- adjustable brightness
- power supply: switchable 110-220 V, 50 Hz

- **31000 BIOLOGICAL MICROSCOPE - 40X to 1000X**
- 30° compensation free binocular head
- eyepiece 10X (Field 18 mm)
- achromatic objectives 4X, 10X, 40X(S), 100X (S, oil)
- semi plan achromatic objective 40X, 100X, 400X, 1000X,
- interpupillary distance adjustment: from 55 to 75 mm
- mechanical stage, double layers (size: 125x135 mm, move range: 75x35 mm)
- Abbe condenser with iris diaphragm and filter
- halogen lamp 6 V, 20 W - brightness adjustment
- power supply: switchable 110-220 V, 50 Hz
• **31005 SPARE BULB for 31000**

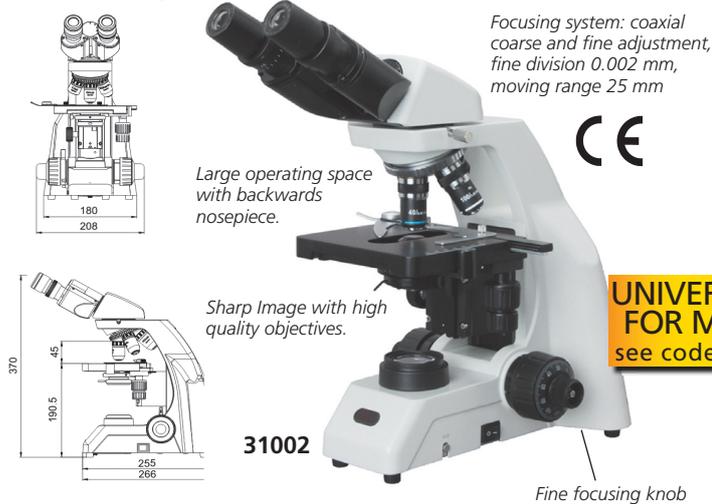

UNIVERSAL CAMERA FOR MICROSCOPES
see code 32185 page 194