

woman' stress-free

Low Level Light Therapy Medical Device For Menstrual Pain

*"Now, Menstrual Pain
Can be Treated!"*

CE₀₁₂₀ / ISO13485
ISO9001 / KFDA / KGMP

color
seven

Treating Menstrual Pain just by

20minutes a day!

Using this device for 20 minutes a day and stop using it at the beginning of your menstruation. The pain can be treated depending on person within 3 months by relaxing the smooth muscles in the uterus tissue cells.




HOW does this device work?

visible-light
irradiation

▶ relaxation of
smooth muscles in
the uterus tissue

▶ increasing
metabolism
of cells


▶ **Menstrual Pain
treatment!**

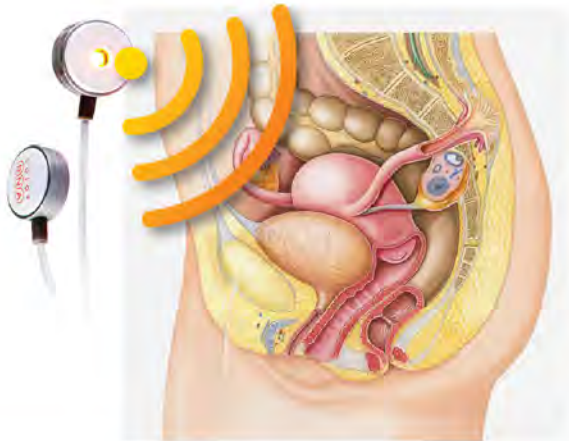

- Visible-light stimulation on your skin arrives at the uterus tissue
- Secretion of Nitric Oxide in the uterus tissue
- Relaxation of smooth muscles in the uterus tissues, which improves circulation of blood
- Treatment of menstrual pain by enhancing the metabolism in the damaged cells

Woman Stress-Free differs in this respect

It was medically verified through clinical tests with Menstrual Pain Patients in two University Hospital. It also showed an outstanding effect!

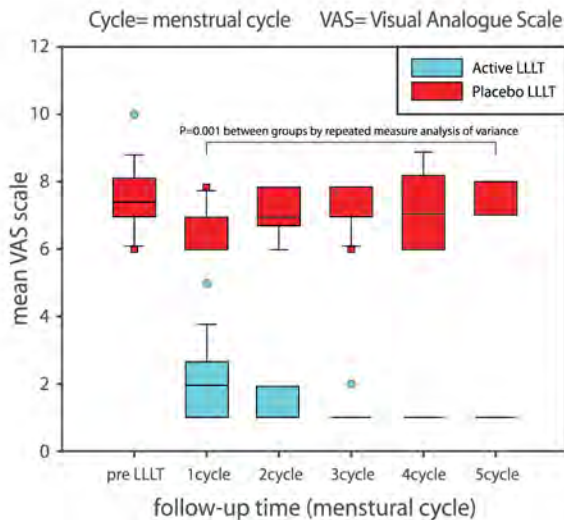

Source ; Archiv Gyneool Obstet(2012) Vol.286:947-952

* A long tradition gynecology & obstetrics journal first published in 1870

According to the graph above the group of people who used the Woman Stress-Free showed a remarkable better result from the first month after the treatment, compared with the group of people who used the Placebo.

The pain was almost completely treated after 3 months of Woman Stress-Free treatment and the patients hardly feel any pain. The group with Placebo menstrual treatment showed a minimal decrease in pain every month.

Woman Stress-Free is a reliable medical device.

Licensed by FDA (Korean Food & Drugs Administration) as a Menstrual Pain Treatment Device,
LLLT (Low Level Light Therapy) Medical device for Menstrual Pain was developed by our independent technology.



HOW to USE Woman Stress-Free (Color DNA-WSF)

This is a Medical Device for Menstrual Pain treatment
Please read the instructions carefully before use



1. Connect the light probe to the main body.



2. Stick the 3M double sided tape on to the light probe matching the hole.



3. Attach one probe on point with a space of two fingers below the navel, and the other on point with a space of two fingers below the first probe in a standing position.



4. Press the main switch to irradiate light for 20 minutes.



* Only use when fully charged

Q & A



Woman Stress-Free (Color DNA-WSF)

Q. *Does 'Woman Stress-Free' really treat Menstrual Pain?*

A. Yes, use 20 minutes a day and stop using the device at the beginning of menstruation. The pain will be treated.

Q. *Was 'Woman Stress-Free' medically verified for safety and effectiveness?*

A. Yes, 'Woman Stress-Free' was clinically examined in two university hospitals for its safety and effects. Consequently, it was licensed by KFDA as a Medical Device for Menstrual Pain.

Q. *How does 'Woman Stress-Free' work?*

A. The visible light stimulation on skin surface reaches the tissues of the uterus and makes the nerve endings release a substance like Nitric Oxide which can induce the relaxation of smooth muscle in the uterus. The relaxation of smooth muscle improve the circulation of blood and provides oxygen and nutrients to the damaged muscle cells of the uterus, which results in the treatment of menstrual pain.

Q. *Is it easy to use?*

A. Yes, it is simple. Stick the double sided 3M tape on the two probes. Attach one probe on point with a space of two fingers below the navel, and the other on point with a space of two fingers below the first probe in a standing position. Then switch on the device.

Q. *Can I use 'Woman Stress-Free' while I'm moving around?*

A. It is an advantage of 'Woman Stress-Free' that we can use it while doing other things, such as watching the television, exercising, and running errands. It is small and light weight so it's easy to carry around.

Q. *Can I use it while I'm on menstruation?*

A. Yes, you may, but the term of your menstruation may become 1-2 days longer so it is better not to use it.

Woman Stress-Free Contents



Main Body



Light Probe



USB Charging Cable



AC/DC Adapter



3M double sided tape
for Medical Use



Instruction Manual



Pouch

Main Body, Light Probe, USB Charging Cable, AC/DC Adapter,
3M double sided tape for Medical use, Instruction Manual, Pouch

color  *seven*

<http://www.color7.co.kr> +82-70-4432-1472 / +82-1577-5691 COLOR SEVEN CO., LTD
4F Geoam Bldg, 62 Banpo-daero 4-gil, Seocho-gu Seoul, 137-867 KOREA