

# Infrarex

## INSTRUCTION MANUAL

<READ INSTRUCTIONS BEFORE USE>

### GENERAL DESCRIPTION

Complete **INFREX** kit includes:

- 1) **INFREX** unit.
- 2) Instruction manual.
- 3) Replacement red lamp bulb.

### SPECIFICATION

Power source : AC 110V / 220V  
Treatment area : 18cm<sup>2</sup>  
Power switch : High / Off / Low  
Weight : 200g

### OPERATION INSTRUCTION

- 1) Insert the plug into 120/220 volt A/C electrical outlet.
- 2) Turn on the unit by sliding the **SWITCH**. The red-colored end (filter cap) will glow red to indicate that the unit is on.
- 3) Place the unit in a horizontal position on a table or other hard surface and allow the unit to warm up for 5 to 10 minutes. **DO NOT** place the unit in a vertical position with the red-colored end (filter cap) face down against the surface of a table, furniture, etc.
- 4) **ALWAYS CHECK** the surface temperature on the red-colored end of the unit ( filter cap ) by lightly and quickly touching the end with a fingertip **BEFORE** applying the unit to any part of the body. If the surface temperature is too warm, then reduce the heat setting by sliding the **SWITCH** to **LOW POSITION**, or else turn the unit off for several minutes. After adjusting the heat setting, **ALWAYS RECHECK** the surface heat temperature in this manner before reapplying.
- 5) The unit has been applied to a particular area of the body about 15-20 minutes each time.
- 6) After finishing, please be sure to shut the unit off and unplug it from the A/C electrical outlet. Store the unit in the protective case along with the instructions and replaced lamp bulb.

### INDICATION FOR USE

The **INFREX** temporarily increases local blood circulation where applied, and temporarily relieves minor muscle, joint aches and pains.

### CAUTIONS


- To avoid electrical shock and possible damage to **INFREX**, **DO NOT** plug the unit into an A/C electrical outlet near a water source such as a bathtub, shower, hot tub, toilet, swimming pool, sink full of water, etc. or near an excessively humid location such as a sauna.
- **DO NOT** use **INFREX** on wet areas of the body. Use on wet areas of the body may cause serious burns.
- **DO NOT** allow children or debilitated individuals to operate the unit without proper supervision, guidance and instruction.
- Temperatures sufficiently high enough to cause burns may occur.
- **DO NOT** use on infants, invalids, or sleeping or unconscious persons.
- **DO NOT** hold or force **INFREX** tightly against the body to the point where the heat or pressure becomes painful. Use **INFREX** before applying liniments and skin ointments.
- **DO NOT** use on areas of the body to which these substances have just been applied. Thoroughly wash and dry any areas to which skin lotions or ointments have been applied before use.
- **DO NOT** use **INFREX** on areas of the body that have recently been sunburned or have received skin or radiation therapy.
- **DO NOT** apply **INFREX** to the head, eyes or other sensitive areas of the body. Consult your doctor or other qualified medical person **BEFORE APPLYING INFREX** on varicose veins, or if in doubt about the appropriateness of applying **INFREX** for a particular purpose.

- **DISCONTINUE USE OF INFREX IMMEDIATELY** if skin shows any signs of discoloration from use of the unit.
- **DO NOT USE IN OR AROUND WATER.**
- **DO NOT** use **INFREX** on abdominal pain. Using on abdominal pain may cause severe complications. If pain persists, consult your physician.
- **DO NOT** use **INFREX** on the area of eyes. Use on this area may cause serious burns.
- **DO NOT** use in areas of poor circulation.

### DIRECTIONS FOR REPLACING THE INFREX LAMP BULB

- 1) **ALWAYS** be sure the unit is **SWITCHED OFF** and un-plugged from the A/C electrical outlet **BEFORE OPENING** the red-colored end of the unit (filter cap) to examine or replace the lamp bulb.
- 2) Grip the handle of the unit in one hand and slowly unscrew (counterclockwise) the red-colored end (filter cap) with the other hand. Next, slowly unscrew (counterclockwise) lamp bulb from the socket.
- 3) Replace the lamp bulb with the extra lamp bulb included in the kit.
- 4) Screw the red-colored end of the unit (filter cap) back on to the unit (clockwise) until it is sufficiently tight. **DO NOT** operate the unit without the red-colored filter cap in place.

### SUGGESTED APPLICATIONS FOR INFREX

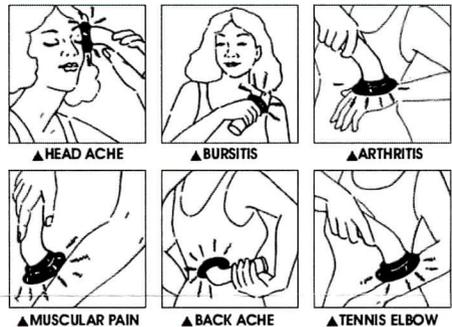

### DIRECTIONS FOR CLEANING THE INFREX UNIT

- 1) Make sure **INFREX** is off and unplugged from the A/C outlet.
- 2) Clean the outside of the unit with a damp cloth or paper towel. Be careful not to allow any moisture to penetrate the seams of the unit. Allow sufficient time to dry by standing the unit in a vertical position against a corner, with the cord end facing down towards a dry towel.
- 3) **DO NOT** immerse the **INFREX** unit in water or a dishwasher.
- 4) **DO NOT** open or dismantle the housing of the unit, as there are no user serviceable parts inside.
- 5) **DO NOT** drop or apply pressure to the outside of the unit, since this may damage or weaken the bulb and other components.
- 6) **DO NOT** use alcohol, benzene or other types of solvent to clean the unit, as such substances will weaken and/or permanently damage the components and materials used in the manufacturing of the unit. If necessary, use only mild dishwashing soap with a damp cloth.

### LIMITED WARRANTY

This **INFREX** (excluding the bulb) is warranted to the original owner to be free from defects in material and workmanship for a period of one year from date of purchase.

This warranty does not cover failures resulting from misuse, abuse and alteration of the device, or failure to comply with operating instructions.



Skylark Device & Systems Co., Ltd  
2F B-9, No. 40-2, Sec 1, Minsheng N. Rd.,  
Guishan Township, Taoyuan County 333, Taiwan  
website: [www.skylarkdevice.com](http://www.skylarkdevice.com)