




Near Vision Card with 16" (40 cm) Cord (#270900)

Lea Numbers Near Vision Card with 16" (40 cm) Measuring Cord and Lea Numbers Pocket NearCard

Instructions

When examining normally sighted children or adults, hold the card at 40cm (16 in), the length of the cord. Let the visually impaired children use their preferred distance and head posture during the first testing, later measure at 40cm (16 in) if the child also uses that distance in visual tasks.

- Start with binocular testing, using the center grouping of symbols.
- Point to each of the four numbers on the top line, observe the baseline responses for comprehension, speed and accuracy.
- Cover the top line with a white card. Make sure that the card does not cause shadow on the line to be read.
- Ask the child/adult to identify only the first number on the line below the covering card.
- Repeat this procedure for each or each other line (moving quickly down the chart to avoid tiring the child) until the child/adult hesitates or misidentifies a symbol.
- Move back up one line and ask the child/adult to identify all the optotypes on that line.
- If the child identifies all numbers correctly go to the next lower line and ask the child/person to identify all the numbers on that line.
- The visual acuity is recorded as the last line on which at least 3 of the 5 optotypes are read correctly. Always test until the threshold line.
- If the chart is held at 16 inches (40cm) the visual acuity value is found in the margin adjacent to that line.
- After binocular testing, proceed with testing each eye separately. In screening, use two pairs of plano glasses for occlusion of the child's eyes or a pair of symmetric glasses that can be used for covering both eyes, one at a time. This is the least disturbing type of occluder.
- For monocular testing, follow the same procedure as for binocular testing.
- Older children and adults may be tested using the reverse side of the near vision card where the numbers are spaced more closely, as if in words or sentences. The close spacing of the symbols on this test makes it a sensitive test for detection of increased crowding effect. In children with brain damage there may be great differences between visual acuity values measured with line test and the more crowded tests. Single symbol acuity may be normal or near normal.
- Visual acuity measured with crowded symbols is closely equal to the smallest text size that the child/person is able to read. It is NOT the size of the texts to be used in learning and at work because nobody likes to read at the level of threshold. We usually read texts that are 3-10 times larger than the threshold size.

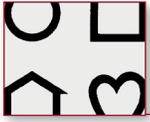

Monocular near vision testing

For monocular testing, follow the same procedure as for binocular testing. As a part of vision screening at school age monocular testing is of interest. If visual acuity at distance has become less than it was before or if there is difference between the two eyes, near vision measurement may give the diagnosis: If the near vision values are symmetric and as before, the change in distance vision cannot be caused by anything else but mild myopia, which does not need to be corrected. Thus the child does not need to be referred - a decrease in the expenses of vision screening and simultaneously an improvement in the quality of screening.

By measuring the near vision values one can follow the development of mild myopia until it starts to affect the child's performance in the classroom, reading from the blackboard, maps and other details at distance. This is the correct time for referral.

If the near vision values are also asymmetric, visual acuity at both distance and near has decreased. In this situation the child needs to be examined by an ophthalmologist to exclude the rare possibility of a non-symptomatic disease. Usually the cause of decrease is a refractive error that may or may not require correction.

Use of near vision testing decreases the number of referrals and at the same time makes screening more sensitive.

Testing Near Vision at Shorter Distances

A child with a visual impairment is allowed to choose any distance and is given a correction for that distance, if needed. If the chart is used at a distance other than the usual 16 inches (40cm), measure and record the viewing distance and the symbol size read (the M value) or the visual acuity value printed adjacent to the threshold line.

To determine the visual acuity use one of the following formulas:


$$VA = \frac{\text{Viewing Distance Used (meters)}}{M\text{-value}}$$

OR

$$VA = \frac{\text{Viewing Distance Used (cm or inches)}}{40 \text{ cm (16 inches)}} \times \frac{VA \text{ value for 40 cm}}{(16 \text{ inches}) \text{ line read}}$$

Note that it is incorrect to report 'V.A. 20/25 at 8 inches' if the child/person could read the 20/25-line (.50M line) at 8 inches. Visual acuity is in that case: $8''/16'' \times 20/25 = 1/2 \times 20/25 = 20/50$. (When using the British notation: 6/9 line at 20cm equals: $20\text{cm}/40\text{cm} \times 6/9 = 1/2 \times 6/9 = 6/18$. When using the decimal notation, the 0.8 line at 20cm equals: $20\text{cm}/40\text{cm} \times 0.8 = 1/2 \times 0.8 = 0.4$.)

When the distance is one half (or one third) of the standard distance, also the visual acuity value is one half (one third) of the value printed next to that line.

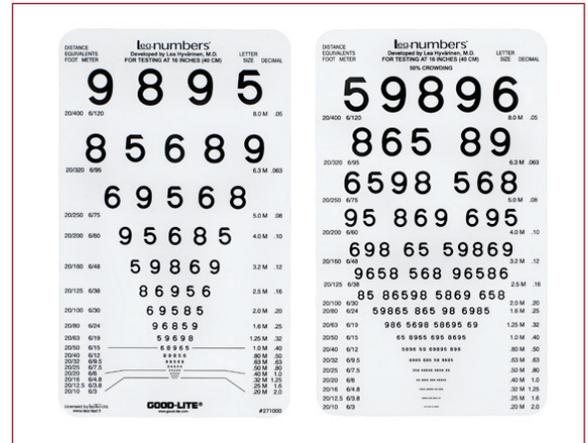

Pocket Near Card (#271000)

20/400	6/120		5 9 8 9 6		8.0 M	.05
20/320	6/95		8 5 6 8 9		6.3 M	.063
20/250	6/75		6 9 5 6 8		5.0 M	.08
20/200	6/60		9 5 6 8 5		4.0 M	.10
20/160	6/48		5 9 8 6 9		3.2 M	.12
20/125	6/38		8 6 9 5 6		2.5 M	.16
20/100	6/30	6 9 5 8 5	6 9 5 8 5	6 9 5 8 5	2.0 M	.20
20/80	6/24	5 9 8 9 6	9 6 8 5 9	5 9 8 9 6	1.6 M	.25
20/63	6/19	8 5 6 8 9	5 9 6 9 8	8 5 6 8 9	1.25 M	.32
20/50	6/15	6 9 5 6 8	6 8 9 6 5	6 9 5 6 8	1.0 M	.40
20/40	6/12	6 9 8 9 5	8 9 8 5 6	9 5 6 8 5	.80 M	.50
20/32	6/9.5	5 8 6 9 6	9 8 5 6 9	5 9 8 6 9	.63 M	.63
20/25	6/7.5	8 6 5 9 8	8 9 6 5 8	8 6 9 5 6	.50 M	.80
20/20	6/6	6 8 6 5 9	5 6 9 8 6	6 9 5 8 5	.40 M	1.0
20/16	6/4.8	9 6 5 9 8	6 5 8 9 8	9 6 8 5 9	.32 M	1.25
20/12.5	6/3.8	8 5 9 6 5	9 6 5 8 9	5 9 6 9 8	.25 M	1.6
20/10	6/3	5 9 5 8 6	8 5 6 9 5	6 8 9 6 5	.20 M	2.0

ANSWER KEY FOR #270900

	50% CROWDING	25% CROWDING	12.5% CROWDING		
20/400 6/120	59896	96568	56986	8.0 M	.05
20/320 6/95	865 89	85 965	69598	6.3 M	.063
20/250 6/75	6598 568	586 8956	968 5968	5.0 M	.08
20/200 6/60	95 869 695	896 968 59	569 685 89	4.0 M	.10
20/160 6/48	698 65 59869	68569 56 896	968 59689 56	3.2 M	.12
20/125 6/38	9658 568 96586	8659 89689 685	6895 869 69586	2.5 M	.16
20/100 6/30	85 86598 5869 658	58596 568 9865 59	8659 568 59685 89	2.0 M	.20
20/80 6/24	59865 865 98 6985	658 6985 89568 58	658 6985 89568 58	1.6 M	.25
20/63 6/19	986 5698 58695 69	5896 965 85 6895	5896 965 85 6895	1.25 M	.32
20/50 6/15	65 8965 695 8695	69 65985 8659 689	956 8956 98 58956	1.0 M	.40
20/40 6/12	5896 59 69895 896	5968 596 98659 56	59 698 5869 86959	.80 M	.50
20/32 6/9.5	8965 685 58 9685	698 59896 95 6985	8569 65985 956 85	.63 M	.63
20/25 6/7.5	958 58596 9658 68	5869 85 98656 986	56896 8956 589 68	.50 M	.80
20/20 6/6	69 8956 958 68959	86 8569 69585 589	985 89659 6985 96	.40 M	1.0
20/16 6/4.8	5869 69856 89 856	95986 859 6598 96	869 6598 58969 65	.32 M	1.25
20/12.5 6/3.8	89569 568 9658 65	658 98 65896 9685	658 5896 89 58696	.25 M	1.6
20/10 6/3	968 6985 58 96598	89569 58 9568 569	59 69585 896 9586	.20 M	2.0

Licensed by lea-test Ltd.
www.lea-test.fi

ANSWER KEY FOR #270900