



# VISIOFOCUS®

Temperature is projected on to patient *WITHOUT* touching



6 in 1

Fever.  
He feels it,  
**YOU SEE IT.**



Made in ITALY by TECNIMED

Also for: objects,  
food, liquids, skin and  
ambient temperature

## FEATURES:

- Projected image indication of correct reading distance.
- Silent, non-contact reader does not disturb your child's sleep.
- Measures and projects the temperature of objects, food, liquids (1 to 55°C) and the skin, as well as the ambient temperature.
- 5 Colour illuminated display.
- 9 Recallable memories.

Model: 06400 displayed



Body Temperature



Bottle



Food



Bath



Bedding



Skin Temperature



Ambient



Memory Recall

# VISIOFOCUS®


Temperature is projected on to patient  
WITHOUT touching



Made in ITALY by TECNIMED

VISIOFOCUS® is the new gentle way to obtain a temperature reading quickly and silently without disturbing your patient. An accurate temperature obtained with a projected reading on the object you have selected (1°C to 55°C) and will work on any surface (food, skin, bottles, liquid etc.). The VISIOFOCUS® will also display the ambient temperature and comes with a 5-colour illuminated display allowing fast recognition of temperature mode. The recallable memory function of the previous 9 temperature readings is at the touch of a button. The simple yet effective aiming system of the VISIOFOCUS® completes the easy-to-use device.

## AIMING SYSTEM

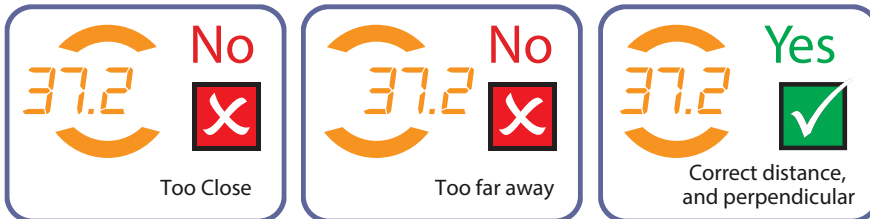




### Temperature Ranges:

- Forehead measuring range: 34-42.5°C
- Other measuring range: 1-55°C
- Ambient temperature working range: 16-40°C \*

\*The devices can work also in environments with a temperature between 5°C and 16°C, but the accuracy and the measuring ranges are not guaranteed.

## QUICK INSTRUCTIONS FOR USE

- Flip open the protective cap.
- Press and hold the "FACE"  button to activate the lights.
- Approach the thermometer perpendicular to the middle of the forehead until the projected temperature comes exactly between the two brackets.
- Release the "FACE"  button holding the thermometer steady until the aiming lights start to blink.
- You can read the value on the display as well.
- The thermometer goes in to standby mode after 20 seconds, displaying the room temperature for 4 hours before turning itself off.
- Carefully read all instructions before use.
- Long life batteries providing 3 years or 10,000 measurements.

Using the "HOME" button , with VisioFocus you can also take the temperature of any object (from 1 to 55°C), such as baby's bottles, soups or the bath's water.

### Accuracy:

#### Forehead measurements

- 36 - 39°C = +/-0.2°C
- 34 - 35.9°C = +/-0.3°C
- 39.1 - 42.5°C = +/-0.3°C

Compliance: ASTM E 1965-98(2009)

#### Other uses

- 1 - 19.9°C = +/-1.0°C
- 42.6 - 55°C = +/-1.0°C
- 20 - 42.5°C = +/-0.3°C

### Specifications:

- Power supply: 4 alkaline batteries type AAA (LR03) 1.5 V (included)
- Battery life: 10,000 measurements or up to 3 years (according to use)
- Dimensions: 144mm x 43,5mm x 21.5mm (protective cap included)
- Weight: 98gm (batteries included)

Distance of operation from the subject fixed through optical signal: about 6 cm (2.36 inch).