



Analyzer

Confiscope G20



Parts Description

- a. LCD display (touch screen)
- b. Power button
- c. Indicator LED
- d. Battery charging port (micro-USB)
- e. Ethernet port (RJ45)
- f. Reset switch
- g. USB ports (2.0 A-type 2EA)
- h. Camera (QR code scanner)
- i. Test device slot
- j. Spare trays



Confiscope G20 Specification

Principle	Colorimetric assay
	Rapid test based colloidal gold
Operating System	Android 5.1.1 (Lollipop) / Linux Kernel 3.4.39
Support Language	English, Chinese
USB Port	USB-A(2.0) 2 Port / Micro-USB 1 Port
Ethernet	Ethernet (802.11 b/g/n)
WIFI Module	2.4Ghz
Adaptor In/Out, Rating	Input : 100~240V, ~50/60Hz, 0.5A Max / Output : 5V/2.0A, Rating : 5V/1.9A
Built-in Battery	Rechargeable Li-ion 5200mAH
Display	5.5-inch color LCD (with Touch)
Light source	White LED (x4): max 3200mW
1st Camera	8M CMOS Camera Module (for Measurement)
2nd Camera	5M CMOS Camera Module (for reading QRcode)
Operating temperature	10 ~ 30°C
Operating humidity	Humidity Max. 70% non condensing
Shipping and storage temperature	-10°C to 40°C
Shipping and storage humidity	Humidity Max. 70% non condensing
Dimensions	19(L) x 10(W) x 11(H) cm
Weight	under 1.3 kg



Serial number for this analyzer.



Manufacturer.



Consult instructions for use.



Date of manufacture:
To indicate the date of manufacture for this analyzer.



Caution, consult accompanying documents.



Do not use if package damaged



Crossed out wheeled bin:
To discard it separately from other household waste.



Symbol for "KEEP DRY".



Authorized representative in the European Community



To indicate that the product is fragile and you need to handle it with care.



Direct current