**seca 233**

**seca**  
Precision for health

Deutsch	Seite 4
Englisch	Page 6
Français	Page 8
Español	Página 10
Italiano	Pagina 12
Dansk	Side 14
Svenska	Sida 16
Norsk	Side 18
Suomi	Sivu 20
Nederlands	Pagina 22
Português	Página 24
Ελληνικά	Σελίδα 26
日本語	28
Polski	Strona 30

Montageanweisung  
Assembly instructions  
Instructions de montage  
Instrucciones de montaje  
Istruzioni per il montaggio

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: [www.seca.com](http://www.seca.com)

## Congratulations!

In the seca 233 measuring rod, you have acquired an accurate and at the same time sturdy piece of equipment for measuring the length of babies.

The measuring rod serves both to determine the length at birth of a newborn and to check body length during the growth process.

It is attached to the following seca baby scales using the assembly material supplied: model 374, 376 and 378

In combination with a scale, therefore, the measuring rod serves as a complete measuring and weighing station.

The measuring rod is manufactured from stable, easy-care aluminium profile. The head and foot supports supplied make measuring convenient and safe.

The seca 233 measuring rod with a seca baby scale are for use in hospitals and doctors' surgeries.

## Safety

Before using the new measuring rod, please take a little time to read the following safety instructions:

- Follow the operating instructions.
- Attach the measuring rod only to the seca baby scales listed above
- Use the assembly material supplied
- When assembling, ensure that screws are tight and that the rod has been assembled correctly (see chapter entitled "Assembly instructions")

- The measuring rod is not a handle. Always take hold of the scale itself if you want to move it.
- When sliding the measuring rod, ensure that neither the baby's hands or feet are on the measuring rod.
- Ensure that the measuring rod and scale are cleaned and disinfected regularly to prevent infectious diseases being transmitted.
- A second person is ideally required to enable length to be measured exactly.

## Assembly instructions

Page 3.

**Note:** the measuring rod is fitted to the tray in elongated holes to allow the measuring rod to be adjusted exactly and unavoidable production tolerances to be compensated. Place a known reference measure between the head and foot stops and move the bracket on the tray so that the correct length is displayed. In general, however, it is sufficient to fit the rod centrally in the elongated holes.

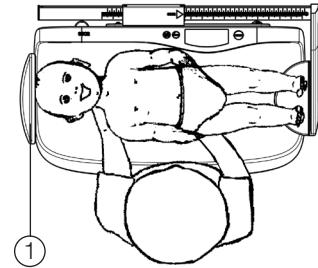
## Operating instructions

### WARNING!


Injury to the baby from a fall. Baby scales are usually placed on elevated work surfaces. If the baby falls from this work surface, he or she may sustain severe, irreversible or fatal injuries.

- Never leave a baby on the scale unsupervised.

### How to measure properly...

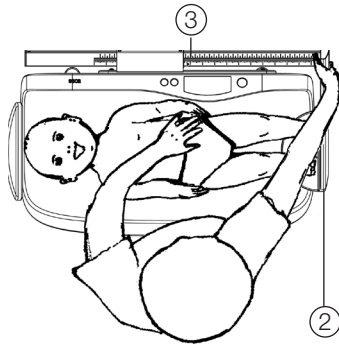1. Lay the baby on the weighing tray. The baby's head must be at the head stop (1) of the tray.

## Technical data

Measurement range: 350-800 mm /  
14 - 32 inches  
Graduations: 1 mm / 1/8 inch  
Accuracy: greater than  $\pm 5$  mm  
Dimensions (W x H x D): 641 mm x 179 mm x  
296 mm  
Weight: approx. 700 g  
Temperature range: +10 °C to +40 °C  
Directive 93/42/EEC  
and Directive 2007/47/EC: Class I with measuring  
function

**CE** 0123

## Disposal

Should the measuring rod no longer be able to be used, the responsible waste disposal association will be glad to give you information about the measures necessary to dispose of it properly.

## Warranty

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts – batteries, cables, mains units, rechargeable batteries etc. – are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage,

claims under warranty can be honoured only if the complete original packaging was used for any transport and the merchandise secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

## CE declaration of conformity

Mechanical equipment for measuring length

seca model 233

Batch no.: see back of measuring rod

The measuring rod satisfies the basic requirements of Directives 93/42/EEC and 2007/47/EC pertaining to medical devices.

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Hamburg, April 2010

**i** : www.seca.com

2. Using your right hand, push the foot support (2) in the direction of the baby's feet, placing the baby's feet on the bearing surface of the foot support. Keep the baby still on the weighing tray with the left hand.

3. With the left hand, gently press the baby's knees down onto the weighing tray and with the right, push the foot support (2) to the soles of the baby's feet. Read off the length of the baby from the marking on the measuring rule guide (3).

## Cleaning

Clean the measuring rod as required using a domestic cleaning agent or commercially-available disinfectant. Follow the manufacturer's instructions.

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