Europa B Evo • Europa B Evo 24

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* Times expressed in minutes (max with cold machine, min. with warm machine).
 The total time includes: vacuum, pre-heating time, cycle and drying phase. The times are to be intended for an autoclave with full loading.

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* Times expressed in minutes (max with cold machine, min. with warm machine).
The total time includes: vacuum, pre-heating time, cycle and drying phase. The times are intended for an autoclave with full loading.

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The Pura osmosis system is an extremely compact, osmosis device designed for connection directly to the autoclave, to guarantee a constant supply. The system can be installed in a number of positions to facilitate mounting and the optimum organisation of the available space. Pura is **fitted with a series of osmotic membranes and filters**, with an average life of between 600 and 800 cycles, depending on the original hardness of the water. The cartridge should therefore need replacing about once a year, or after 800 cycles. **Changing the filter is an extremely simple process** and can easily be carried out by the users. **The overall cost of the filters is very reasonable**, and the cost per litre is therefore almost negligible. When the Pura system is not producing water, it is not pressurized, and this avoids all kinds of problems caused by the accidental breakage of couplings and fittings.

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The only autoclave with integrated biological test management

Periodically testing autoclave efficiency is a requirement of laws and regulations in force.

Tests include a biological test, which determines whether an autoclave can eliminate any life and pathogenic form. Tecno-Gaz has fitted its Europa B Evo autoclave with a biological incubator connection and management system. This system, which is new worldwide, lets operators connect the B TEST device to the autoclave, independently managing operation. The B TEST device has 3 positions. It connects to the attachment on the autoclave and can be used to manage the biological incubation cycle of the phial. Although directly operated by the autoclave, this device does not interfere with autoclave functions and is even enabled when the autoclave is off, as it can be operated autonomously and individually.

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The main commitment of a company like Tecno-Gaz is to manufacture products with unique features, distinguishing traits and concepts that ensure maximum reliability and safety to the user.

Our research offices, together with support and assistance of external institutions have devised an extraordinary system that has been applied for the first time to our autoclaves. This innovative device is called P.I.D. P.I.D. means Proportional Integral Derivate and represents the apex of closed loop controls. The structure of this technologically advanced control is simple and flexible. P.I.D. ensures a constant control and monitoring of the whole operating system of the autoclave, acting automatically in case of anomalies or incorrect values in technical and electronic parameters. This device behaves like a "real technician" with real-time diagnostic and intervention capability. This solution reduces anomalies and warnings, granting maximum reliability and minimising the machine inactivity times (if any).

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The first goal for a company like us is to produce “flexible” devices to suit the needs of all operators, to facilitate the machine operation and to allow saving of time and money. During project development our technicians’ top priority is therefore to devise new technical concepts and solutions that may grant our products flexibility, which is of the utmost importance. Autoclaves Europa B Evo and Europa B Evo 24 can rely upon a number of solutions that make it possible for the operator to pre-set the autoclave according to his/her operating requirements.

Loading water tanks: set the autoclaves according to your needs

You can choose how to pre-set Your autoclave.

The Europa B Evo autoclaves is provided ready for:

- working autonomously, using water in the inner tanks and operating loading and manual unloading of pure and used water.
- be connected directly to the water supply network by using the Deioniser (Pura model), thus allowing You to have automatic water supply and discharge without losing any time in manual operations.

Advice

If You use the autoclave constantly but do not want to face the problems usually related to purchase of distilled water and handling tanks loading and unloading we recommend You to use the system equipped with Pura model deioniser this way you will save time and money while protecting your autoclave thanks to an unending supply of water, guaranteed 100% pure. The autoclave Europa B Evo24 is already equipped with the Pura device.

Internal tanks

All the autoclaves are equipped with an internal tank for pure water and another for used water. Each tank is equipped with probes to detect the minimum and maximum water levels.

Tank filling

If the autoclave is connected with the Pura device the tanks are automatically filled, but if you want to use it autonomously loading shall be carried out manually. Tecno-Gaz autoclaves, to facilitate filling operation, are equipped with a pump-operated filling system featuring a level sensor that stops water filling if the tank is full. This useful option is included on all the models of the line. It avoids the risk of human errors due to excess pouring and it also reduces operator efforts. The autoclave is preset to allow manual filling of the tank in case the pump is out of order.

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Sterilization cycles

Preset temperatures

The preset temperatures used in our autoclaves' cycles are 134°C and 121°C. 134°C cycles are normally used for sterilization of solid and ferrous materials. 121°C cycles are normally used for sterilization of thermoplastic and delicate materials. In sterilizing materials, always follow the recommendations of the device manufacturer.

Ease of use

Choosing and selecting a sterilization cycle on Tecno-Gaz autoclaves is extremely simple and intuitive. One of the buttons makes it possible to scroll and view data about all sterilization cycles on the display, thus allowing You to find the cycle that suits your materials best, then You will only have to push another button to automatically start the whole sterilization cycle without having to worry about anything. In case of any error or warning a series of messages and LEDS will attract your attention to the problem.

Fully automatic cycles

All cycles are fully automatic, with all steps managed by the leading-edge computer board with a powerful microprocessor. Each step (vacuum, preheating, exposure time, drying) is automatically monitored, assisted and handled. There is no need for human intervention in cycle management.

Wide choice of operation cycles

The Europa B Evo and Europa B Evo 24 autoclaves are equipped with:

- 2 sterilization cycles, temperature 134°C for free or packed materials.
- 2 sterilization cycles, temperature 121°C for free or packed materials.
- 1 "flash" cycle for quick sterilization cycles.
- 1 "prion" cycle especially designed for Cruetzenfeld & Jacobs disease.

Cycles without operator intervention and with auto-shutdown

All sterilization cycles have an automatic function that, if the operator is not by the machine when the cycle ends, will trigger

an anti-condensation process and auto-shutdown operation. This system makes it possible to economize and increase safety of the device.

Test cycles

Our autoclaves are equipped with test cycles for periodic control of the device by means of physical and functional tests.

Helix test to evaluate vacuum capability in porous bodies. Bowie & dick test to evaluate vacuum capability in hollow bodies.

Vacuum test to evaluate vacuum retaining capability.

Constantly monitored cycles

The autoclaves Europa B Evo - Europa B Evo 24 are equipped with a device used to constantly control and monitor the sterilization cycles. Any anomaly or uncommon feature is quickly detected and, if it jeopardises the sterilization cycles, immediately blocked. The operator is then warned of the anomaly by a message.

All cycles ensure sterilization of b-type loads

All the sterilization cycles feature the fractional vacuum system as well as controls and parameters that ensure sterilization of B-type loads. Therefore, even if the type of cycle chosen is incorrect the sterilization carried out on the material will be equally adequate.

Ergonomic structure and ease of use

The sterilization cycle is chosen by pushing one button and activated by pushing another button.

It is impossible to make a mistake or to choose a cycle that provides inadequate sterilization, all operational errors are notified by error messages.

The autoclave can be used by more than one person without risking to cause problems or damages. Tecno-Gaz wanted to provide maximum operation safety even on this important issue.

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Chamber and load

THE TECNO-GAZ AUTOCLAVES ARE ALL BUILT WITH A SPECIAL STAINLESS STEEL. THEY ARE MANUFACTURED WITH LEADING-EDGE INDUSTRIAL TECHNIQUES, THEY UNDERGO STRICT CHECKING PROCEDURES AND TESTED ONE BY ONE. THEY ARE T PED CERTIFIED, COME PREVISTO DALLA DIRETTIVA PED.

Load and size

The sterilisation load capacity of a device is an extremely important parameter to consider when choosing an autoclave.

The size of chambers is very similar, however the load sterilisation capacity of each model varies considerably. In fact this parameter is dependent on technical choices, such as the type of steel used to construct the chamber, the heating system adopted, etc...

The Europa B Evo autoclaves can be used to sterilise significant loads:

- Europa B Evo , instrument load up to a maximum of 4 kg
porous materials up to 1,5 kg
- Europa B Evo 24 , instrument load up to a maximum of 6 kg
porous materials up to 2 kg

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Safety on the door

We have applied two safety devices on the autoclave door:

- The first device is a mechanic and electro-magnetic one: it triggers when the sterilization cycle begins and completely blocks the door opening. This device can be removed at the end of the cycle by pushing the device-unlocking button, but only if positive pressure is near zero.
- The second device is an electrical one: it triggers automatically when an attempt to force the door open is detected.

Safety devices in place

- Certified safety valve
- Ped on sterilization chambers
- Insulation of the chamber with certified ceramic material.

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Tecno-Gaz uses all the standards required by safety legislation in force. But in addition to that we have decided to comply with these regulations adding also other technical solutions for our customers.

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Close-up view of the front of an Europa B Evo autoclave

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A rich equipment supply

We think that each autoclave shall be equipped with the tools and instruments that are used to successfully carry out everyday work as well as those accessories necessary to control and manage the device itself.

This is why Tecno-Gaz autoclaves come already equipped with all available accessories and presets.

- **Built-in printer:** the printer ensures recording of sterilization cycles' data and production of a formal vacuum test certificate.
- **Memory card connection:** the machine can be connected to a Tecno-Gaz Memory Card
- **Water distribution network connection:** all the autoclaves are equipped with a connection to the water distribution network with a Pura Deioniser.
- **Other accessories:** 1 closed tray holder, 4 aluminium tray holders, 1 double-use key for tray extraction and door adjustment, water loading and unloading hoses, chamber cleaning sponge.

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Hydraulic connections and fitting for
Pura deioniser

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Integrated biological incubator mod.B Test

Three-phial biological incubator, to connect directly to the Europa B Evo

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Deionizer Pura model

- Deioniser equipped with electronic control unit for connection and communication with the autoclave.
- The cartridge filter has a product life of about 800 cycles.
- This device helps saving time in the everyday working activity and cost on the purchase of distilled water.

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Pc link software

- Dedicated software for data acquisition and management with a pc.

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Memory card plus software

- Easy-to-connect memory card to acquire data about the performed cycles.
- Allows storage of up to 90 cycles.
- Equipped with software to download on the pc the cycles stored in the memory card.

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Special double-use tray holder

- Special tray holder designed to hold 4 standard open trays and 3 trays with cover.

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A correct sterilization, using the correct instruments, is useful in order to protect oneself from damages compensation claims.

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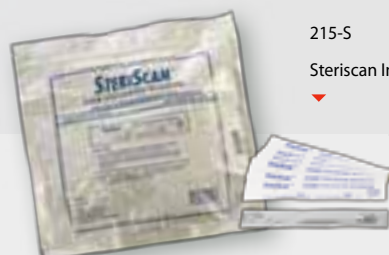
* The specifications given above are indicative. The operator must comply with regional or national laws/standards regulating sterilization.

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All rights reserved. Variations can be done without notice. Tecno-Gaz S.p.A. is to be considered not responsible for damages caused by the lack or the wrongness of the information here mentioned.